PT Gaze

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[EN] Gauze

DESCRIPTION HYDROCYN aqua Gauze helps to clean wounds. The dressing also increases the moisture within the wound as the gauze is soaked with HYDROCYN aqua Solution. It makes it effective in assisting the debridement and desloughing process in dry necrotic wounds. It helps by removing dead, damaged, or infected tissue to improve the wound healing process.

INGREDIENTS Hypochlorous Acid (0.003%), Sodium Hypochlorite and Sodium Chloride (≤0.1%), Purified Water (99.9%) and Woven Gauze.

INDICATION HYDROCYN aqua Gauze is intended for mechanical protection of the wound, cleaning, debridement and desloughing of ulcerated wounds, burns, post-surgery wounds and bed sores. It is also used as a primary or secondary dressing.

DIRECTION FOR USE

- Place HYDROCYN aqua Gauze on the wound bed for 10 to 15 minutes.
- After soaking, remove the gauze and swab to clean/debride the wound area using the gauze.
- Then, the wound can be dressed with HYDROCYN aqua Gauze or other type of dressing of your choice.
- Change dressing once a day or as indicated by the clinical condition of the wound. Maintain a moist wound environment between dressing changes.

STORAGE CONDITION HYDROCYN aqua Gauze should be stored in a cool, dry place (recommended storage temperature is 4°C to 30°C) away from direct sunlight/extreme weather condition, in its original packaging.

STABILITY Product is stable for 18 months from the date of manufacture if unopened and in its original packaging. For single use only.

CAUTION Do not use the product if the seal is damaged or tampered with. Packaging must be discarded if signs of contaminants are seen. Do not use the product in combination with other topical creams unless with the advice of a health care specialist. Consult your health care specialist if wound persists to deteriorate even after use of the dressing. Notify your health care specialist in case of any adverse effect. This device is intended for "Single Use".

RISK OF RE-USE HYDROCYN aqua Gauze cannot be re-used if it is removed from the packaging.

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