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CHILDREN'S ORTHOPEDIC FOOTWEAR

READ THE INSTRUCTIONS CAREFULLY BEFORE USING THE MEDICAL DEVICE!

WARNING: Keep the packaging for traceability and identification of the medical device throughout its lifetime!

Dear customers,

Thank you for purchasing the footwear distributed by our company.

Children's orthopedic footwear is the medical device of class I, which is supplied to the market unsterilized and is not intended for measurement purposes..

Intended purpose

Children's orthopedic footwear is intended for the treatment of orthopedic foot defects.

Indications

- varus,
- valgus,
- equinus.

Contraindications

- the medical device must not be applied to open wounds.

Features

Children's orthopedic footwear are made of natural upper materials. They are produced in groups according to the season:

- summer,
- all-season,
- winter.

The height of the rear parts of the footwear and the reinforcement ensures the correct and firm position of the ankle, and thus the foot when walking.

The inner space of the footwear is filled with a softened, coated insole with corrective elements. The heel part of the upper of all models is reinforced with a thermoplastic heel and side reinforcement parts, which are inserted under the upper insert of the rear parts. The upper of the footwear is closed with lacing or Velcro tape. The upper collar is reinforced.. Children's orthopedic footwear is produced in the size range from No. 20 to 46 of the French numbering - depending on the specific model.

Correct use

Children's orthopedic footwear is intended for walking purpose; it is not suitable for sports and tourist activities.

In the beginning of using orthopedic footwear, gradual getting used is recommended.

When choosing the cut solution for the upper and the construction of the bottom of the footwear, please pay attention to the recommendation of a specialist.

The footwear is not suitable for use on bare feet; suitable socks contribute to easier absorption of sweat and prevent pressure and damage of the skin.

Do not use the footwear if your feet are injured or swollen.

The footwear is intended for use by a single person and are not transferable.

Do not interfere with the construction of the footwear, as this could lead to a deterioration in the health of your feet or even injury.

Residual risks, warnings

R-Loss of effectiveness of the device, possible deterioration of the user's health

- The footwear must be prescribed by a competent specialist and/or consulted with an orthopedic technician,
- the time and method of using the footwear will be determined by the doctor,
- the design of the product must not be interfered with,

- the loss of effectiveness of the footwear, or the deterioration of the patient's health may be affected by storing or drying the product near heaters and open flames at a distance of less than 1 m.

R- Occurrence of skin allergy, infection

- The product is intended for repeated use by one person only and is non-transferable,
- care must be taken that the device does not exert pressure on those parts of the body that have been injured or swollen.

R-Incorrect size selection

- a smaller size of the footwear can cause strong contraction in the applied area with subsequent lack of blood flow, with a large size there is a risk of loss of the therapeutic effect of the product.

R-when used on open wounds, the development of superficial or internal infection

- the footwear must not be applied to open wounds.

Handling, care, storage and disposal of the product

- the footwear made of natural leather - uneven surface pattern, partially different color shades are characteristic of natural leather. With increased sweating or getting wet, the footwear may partially bleed color. Proper and careful care extends its lifespan.
- the footwear made of smooth leather – clean mud and dust with a damp cloth and treat with a suitable cream,
- the footwear made of hair leather – clean dirt with a rubber brush. To remove grease, use a chemical stain remover. Treat with hair leather paint. Never cream footwear made of the above materials!
- if the footwear is damp, it is necessary to dry it, but not in contact with or in the immediate vicinity of a heat source,
- we do not recommend : machine wash, dry clean, bleach with active chlorine-containing agents, tumble dry,
- store the product in a dry environment,
- use mixed waste when disposing of the product.

Warranty

- the warranty period is 24 months from the date of sale of the products (the warranty period cannot be confused with the service life of the device),

Complaints

- the complaint applies only to material defects or manufacturing defects. In the event of a complaint about products, the buyer is obliged to submit proof of purchase of the products, and when issuing a voucher for a medical device - a warranty certificate.
- serious accidents caused by this medical device must be reported to the manufacturer.

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The company PROTETIKA,a.s. meets the requirements of ISO 13485:2016
The product complies with the requirements of Regulation (EU) 2017/745.

Date of issue of the instruction manual: 12/2024