
FORLIFE
BERLIN

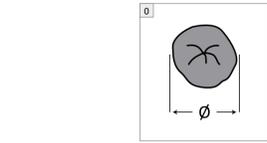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STOMOCUR® CE

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EN Instructions for use

Please read the instructions for use carefully, as they contain important information on how to use your STOMOCUR® products.

One-piece urostomy system

Intended purpose: Urostomy pouches are designed for people with an artificial bladder outlet. Urostomy pouches are used for the collection of urine and are designed to be single-use. They have a lockable outlet valve through which the collected urine can be emptied.

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Preparation

Wash your hands carefully before and after each use.

If the hole opening of the system is too large, the skin is not covered sufficiently. Skin irritation and inflammation can occur owing to long periods of contact with stoma material. You should, therefore, determine the diameter of your stoma before use (Fig. 0).

Our products are also available in pre-cut hole sizes.

Note: Use **STOMOCUR®** **Fast Rings** for extra adhesive surface. You can use **STOMOCUR® Skin Protection Paste** or **Skin Protection Rings** to smooth out unevenness and avoid skin problems.

Preparation

Cut the opening of the skin protection plate to fit the size of your stoma using a pair of scissors (Fig. 1).

The turn valve is set in the closed position at the factory (Fig. 2b). To avoid inconvenience, please close the valve with the sealing cap.

After connecting the urine collection pouch, open the turn valve 180° clockwise to ensure flow.

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Preparation

Cut the opening of the skin protection plate to fit the size of your stoma using a pair of scissors (Fig. 2b). To avoid inconvenience, please close the valve with the sealing cap.

After connecting the urine collection pouch, open the turn valve 180° clockwise to ensure flow.

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