AT52325, AT52326

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EN

Thank you for purchasing our product, please read the instructions for use carefully. Do not use the product until you have read and understood these instructions. If you do not understand the warnings, cautions and recommendations, please contact your healthcare professional or dealer to avoid damage to the product.

NOTE: Inspect all parts for damage caused in transit. If such damage is noticed, DO NOT use the product. Contact the manufacturer for more information.

APPLICATION

The product is designed for the mobility of disabled people with limited or no walking ability at all. The wheelchair is designed to transport one person.

INDICATIONS

-Severe **limitations of movement:** people with severe limitations of muscle strength, for example due to neurological diseases, muscular diseases or pain syndromes, which prevent or significantly hinder independent movement.

-**Neurodegenerative diseases:** People with diseases such as multiple sclerosis, Parkinson's disease, muscular dystrophies or ALS, which lead to progressive loss of muscle control and strength.

-**Fatigue:** People for whom prolonged movement or manual handling of a wheelchair is too much of a burden due to illness or medical condition, causing rapid fatigue and exhaustion.

-**Age-related limitations:** Older people who have limited mobility and strength due to age can use an electric wheelchair to maintain their independence and mobility.

-**Rehabilitation and recovery periods:** People in rehabilitation after injury or surgery who are temporarily unable to move independently can use an electric wheelchair as part of their treatment and recovery process.

CONTRAINDICATIONS

Physical or mental limitations (e.g. visual impairment) that prevent safe handling.

-Perceptual impairment: Individuals with difficulty judging distance or space may have difficulty maneuvering an electric wheelchair safely.

-Inability to operate controls: Electric wheelchairs require a certain amount of manual skill and coordination to operate a joystick or other controls. Individuals who cannot effectively operate these devices may not be able to safely use an electric wheelchair.

-Severe cognitive impairment: Although many people with cognitive impairment can safely use electric wheelchairs, for some, severe impairment may prevent safe driving.

-Postural instability: People who cannot maintain a stable sitting position may require additional support systems to safely use an electric wheelchair.

-Health conditions that affect consciousness or reactions: For example, epilepsy or other conditions that can lead to sudden deterioration of consciousness or reactions may be a contraindication.

FEATURES AT52325

-carbon frame covering

-swivel armrests

-adjustable footrest length

- parking brakes

- special locking device to prevent the frame from folding

-antitipping system

-ergonomic seat

- grip for carrying the folded wheelchair
- motors built into the wheels
- 4 dampers

FEATURES AT52326

- carbon frame covering
- swivel armrests
- adjustable footrest length
- parking brakes
- special locking device to prevent the frame from folding
- antitipping system
- ergonomic seat
- grip for carrying the folded wheelchair

SCOPE OF SUPPLY

- wheelchair
- charger
- accumulator
- set of adjustment spanners
- safety belt
- operating instructions

NOTES:

In the event of a product-related 'serious incident' that directly or indirectly led to, could have led to or is likely to lead to any of the following:

- (a) the death of a patient, user or other person; or
- (b) temporary or permanent deterioration of the health of a patient, user or other person, or
- (c) a serious threat to public health

this 'serious incident' should be reported to the manufacturer and to the competent authority of the Member State in which the user or patient is resident. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

NOTES:

If you experience pain, allergic reactions or other distressing, unclear symptoms related to the use of the medical device, consult a healthcare professional.

NOTE: it is forbidden to use the product other than for its intended purpose .

ATTENTION: The manufacturer will not be held responsible for damage caused by negligent maintenance of the product, use of the product contrary to the instructions, or resulting from failure to observe the notices contained in this manual.

GENERAL WARNINGS

- When getting in or out of the wheelchair, make sure that both brakes are on.
- Steep and uneven surfaces should be avoided, as this can cause the product to tip over.
- Avoid soft surfaces as the wheels may collapse.
- Move the wheelchair smoothly in all conditions.
- Always observe your surroundings to identify hazards, such as furniture, doors, holes, hills, kerbs, entrances, etc.
- Keep both feet on the footrests at all times, and ensure that clothing is properly tucked and does not block the wheels.
- Always drive carefully on hills.
- Always keep both hands on the push handles for proper control of the wheelchair (applies to the carer).
- Always remember to maintain balance.
- Keep the wheelchair away from sources of fire.
- Under no circumstances should the maximum load on the product be exceeded.
- Do not hang any heavy objects on the back of the wheelchair , as this may affect the stability of the product especially when climbing hills.
- Only one person can use the wheelchair at a time.
- Do not use a wheelchair when the patient is unconscious.
- Do not use other means of transport to push the wheelchair.
- Do not lift the front wheels while driving, as this may cause a fall and serious injury to the user.
- Do not get into a wheelchair while moving.
- Do not drive the wheelchair too fast and do not make sudden turns.
- Do not drive a wheelchair across a hill (in either direction) with a gradient of more than 7 degrees.
- Do not stand on the footrests when getting on or off the product.
- Do not negotiate stairs using a wheelchair.
- Do not attempt to lift the wheelchair with moving (detachable) parts. Lifting with moving parts of the product may lead to injury to the user and/or carer or damage to the product.
- Do not get the product wet.

TARGET PATIENT GROUP

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in these instructions for use). The device can be purchased by the user independently or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

TECHNICAL SPECIFICATIONS AT52325

Dimensions: 106*55*92 cm
Dimensions when folded: 55*37*78cm
Seat depth: 40 cm
Seat width: 42 cm
Height from ground to seat: 51 cm
Backrest height: 45 cm
PU front wheel (solid): 7"
PU rear wheel (solid): 8"
Weight: 19.4 kg
Max load: 120 kg
Pack size: 57.5*40.5*84 cm
Adjustable footrest length: 4 steps in 2.5 cm increments
10A/6A-20A lithium battery (quick and easy connection)
Brushless motor: 250/W024V
Rated voltage: 25,2 V
Rechargeable lithium-ion
Max speed: 6 km/h
Range (6A): 9 km
Range (10A): 15 km

TECHNICAL SPECIFICATIONS AT52326

Dimensions: 106*55*92 cm
Seat depth: 40 cm
Seat width: 42 cm
Height from ground to seat: 51 cm
Backrest height: 45 cm
Height of armrest: 24 cm
Adjustable footrest length: 4 steps in 2.5 cm increments
Honeycomb front wheel (solid): 7"
Honeycomb rear wheel (solid): 9"
Weight: 13.9 kg
Max load: 100 kg
Pack size: 60*37*83 cm
10A/6A-20A lithium battery (quick and easy connection)
Brush motor : 180W/24V
Rated voltage: 25,2 V
Rechargeable lithium-ion
Max speed: 6 km/h
Range (6A): 9 km
Range (10A): 15 km

BATTERY OPERATING INSTRUCTIONS

- 1: Charge the battery only at temperatures from 0 to 40 degrees Celsius. Use only at temperatures from minus 20 to 45 degrees Celsius. Keep the battery dry.
- 2: It is forbidden to place the battery in acidic, alkaline, corrosive liquids or water.
- 3: It is strictly forbidden to place the battery in a high-temperature environment near a source of heat. Do not expose the battery to direct sunlight.
- 4: Any action which goes beyond the normal use of the battery is strictly prohibited, including: deliberate damaging of the battery, disassembly, etc.
- 5: It is recommended to use a dedicated original charger, do not charge for more than 7-10 hours, the charging current should not exceed 2A. The battery should always be fully charged.
- 6: If the battery is not used for a long period of time, store it in a cool, dry place and recharge it every 2 months.
- 7: To charge the battery, connect the charger output plug to the charging socket on the back of the controller
- 8: When the charging indicator on the charger lights up green, this means that the battery is fully charged (the capacity of the battery is 10 Ah and the charging time depends on the amount of energy remaining in the original battery.) An uncharged new battery will need to be charged for approximately 4-5 hours the first time). After charging, disconnect the charger's power plug and the output plug.

Charging method 1: insert the charger plug into the charging port at the bottom of the remote control. Note the direction of connection of the charging plug (as shown in the figure below).

Charging method 2: direct charging of the battery. There is a charging port on the battery housing. The charger can be connected directly to the battery (as shown in the figure below).


SYMBOLS

	Manufacturer
	Attestation of conformity of the product with the essential requirements of EU regulations
	Medical device
	Lot number
	Date of production
	Please read the instruction manual before use
	Warning
	This symbol indicates the maximum weight of the user

DISPLAY

- 1.Before unfolding the frame, release the safety lock by pressing the red button (fig.1)
- 2.Then unfold the frame by pulling the seat and push handle outwards until fully extended. To ensure that the wheelchair is correctly unfolded, press down on the seat.
- 3.Fold out the footrests and adjust their length using the length stop (fig.4)
- 4.Then unfold the armrests
- 5.Finally, install the controller according to the following instructions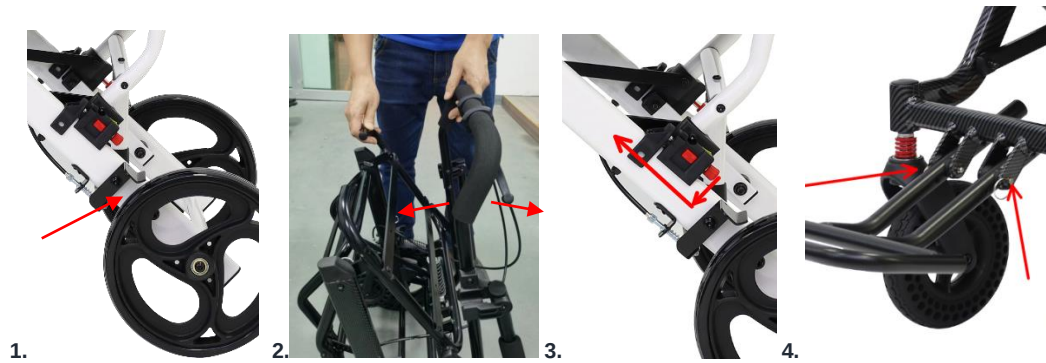

SUBMISSION


To assemble the product, all of the above steps must be performed in reverse order.

INSTALLATION OF THE CONTROLLER



The controller is by default positioned on the right side. First loosen the screw. Then insert the rod into the slot of the positioning bracket, then move to the correct position if necessary and tighten the screw. Finally, connect the controller cable to the battery connector and tighten the coupling nut.(as shown in the image below).

When the connector is docked, note the alignment of the groove. The black arrows on the left and right connectors should match. To avoid damaging the connector, do not insert it by force without matching the grooves.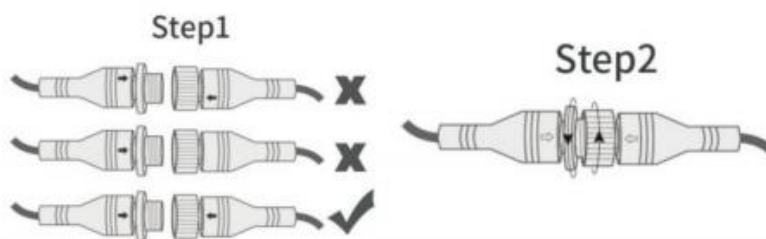

USE OF THE AT52325 CONTROLLER

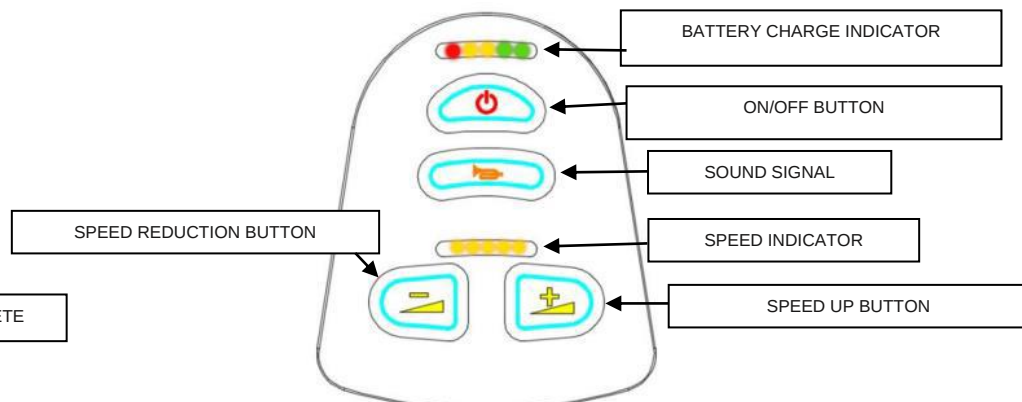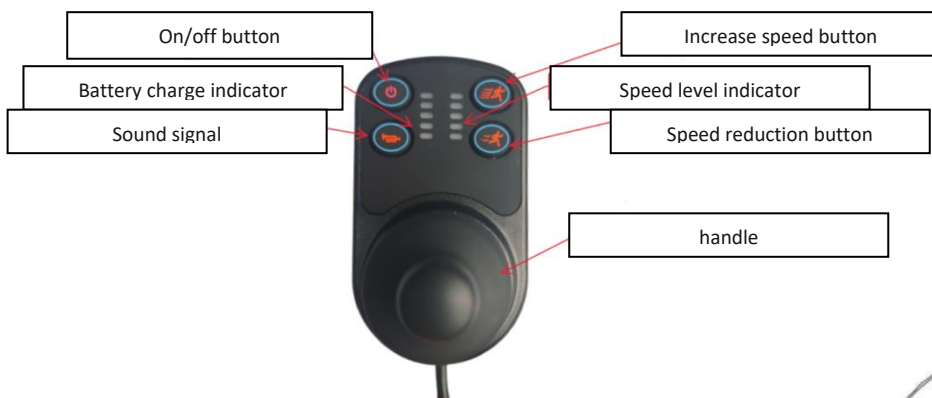On/off button

The button is responsible for controlling the power supply to the controller electronics and for controlling the power supply to the motor. Do not use the on/off button to stop the wheelchair except in emergency situations. Using the button for this purpose may adversely affect the life of the product.

Control lever

The handle is used to control the speed and direction of the wheelchair. Speed is controlled by the degree of deflection of the handle relative to the central axis. When the handle is released, the control returns to the centre position and the brake is automatically applied.


TRANSPORT

The trolley is equipped with an additional function for transport.

BACKGROUND

To tilt the armrest, press the red button and lift the armrest.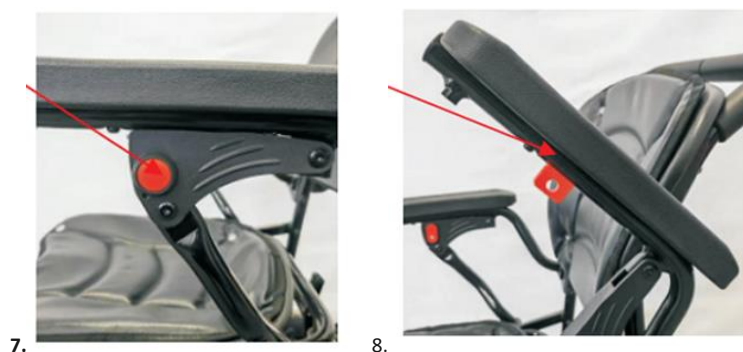

BRAKES

To apply the brake, push the brake rod forward until the wheel locks. To unlock, release the brake rod.

BRAKING SYSTEM

1. the wheelchair supports two driving modes: electric and manual. When the controller is switched on, the electric mode of the wheelchair is automatically activated. When the controller is switched off, the wheelchair switches to manual driving mode. It is important to ensure that both brakes are always on when the wheelchair is stationary.
2. When driving the wheelchair, if the screen shows a power output of less than 10%, the system will automatically switch to speed limit mode (the speed will be reduced to less than 1 km/h). In this situation, the electric wheelchair can still drive about 1 kilometre. If the power drops below 2%, an automatic stop will occur and the message "Driving Ban" will appear on the screen. In this situation, it is advisable to tighten the rear wheel brakes or switch off the remote control to switch to manual driving mode.
3. when driving on uneven or bumpy roads, it is recommended to use manual mode without activating the electric drive system.

CLEANING AND MAINTENANCE

1. SEAT AND BACK: to clean the seat or backrest, unscrew the screws and remove the fabric element.

13.

2. OTHER ELEMENTS:

Clean with a cloth and mild detergent. When cleaning, take care not to use excessive force. Do not use abrasives or alcohol-based cleaners, rough objects or solvents.

STORAGE

Store the product in a dry, cool room, away from moisture and direct sunlight. .

HOW TO DISPOSE OF THE PRODUCT

Dispose of the electrical medical device in accordance with the obligatory rules. Handle in a manner appropriate to the disposal of electrical and electronic equipment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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