INSTRUKCJA UŻYTKOWANIA
AT51130/AT51131 KULA ŁOKCIOWA
USER MANUAL
AT51130/AT51131 ELBOW CRUTCH
GEBRAUCHSANWEISUNG
AT51130/AT51131 ELLBOGENKRÜCKE
NÁVOD K POUŽITÍ
AT51130/AT51131 LOKETNÍ BERLE
NÁVOD NA POUŽITIE
AT51130/AT51131 LAKTEĽNÁ BARLA
MODE D'EMPLOI
AT51130/AT51131 BÉQUILLE D'AVANT-BRAS

MD

PL

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En

Thank you for purchasing our product. Please carefully read the user manual. Do not use the product before reading and understanding this manual. If you do not understand the warnings, notes, and recommendations, contact a healthcare specialist or the seller to avoid product damage or personal injury.

WARNING: Check all parts for damage that may have occurred during transport. If any damage is found, DO NOT use the product. Contact the seller for more information.

Before using any mobility aid, consult your doctor or specialist who can provide advice and instructions on the most appropriate way to walk, climb stairs, stand up from a chair, or sit down with elbow crutches.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for which this product is intended for treatment, rehabilitation, or compensation (see the product purpose section in this manual). The product can be purchased independently by the user or based on the recommendations of a doctor, therapist, or other specialist. Whether purchasing independently or upon medical recommendation, the user should consider available sizes, necessary functions, product variants, indications and contraindications for use, as well as information provided by the manufacturer.

APPLICATION/INDICATIONS

Elbow crutches are a walking aid. They are designed for individuals aged 12 and older (teenagers and adults) with a maximum weight of 120 kg who have limited walking ability. They should not be used for any purpose other than walking support. The product can be used both indoors and outdoors.

CONTRAINDICATIONS

Physical or mental limitations (e.g., visual impairment) that prevent the safe handling of the product.

TECHNICAL SPECIFICATIONS AT51130

Durable aluminum tubes

Ergonomically shaped handle

High-quality soft rubber tip

Hard grip

Distance handle – forearm support: **235 ± 2 mm**

Distance handle – ground min.: **765 (895) ± 10 mm**

Distance handle – ground max.: **990 (1 120) ± 10 mm**

Weight: **600 (620) g ± 5**

Crutch length min.: **970 (1 105) ± 10 mm**

Crutch length max.: **1 195 (1 330) ± 10 mm**

Maximum load capacity: 150 kg

THIS SYMBOL INDICATES THE MAXIMUM USER WEIGHT.

TECHNICAL SPECIFICATIONS AT51131

Durable aluminum tubes

Ergonomically shaped handle

High-quality soft rubber tip

Soft grip

Crutch length min.: **970 ± 10 mm**

Crutch length max.: **1 195 ± 10 mm**

Distance handle – forearm support: **235 ± 2 mm**

Distance handle – ground min.: **765 ± 10 mm**

Distance handle – ground max.: **990 ± 10 mm**

Weight: **600 g ± 5%**

Maximum load: **150 kg**

THIS SYMBOL INDICATES THE MAXIMUM USER WEIGHT.

COMPONENTS

Elbow crutch, User manual

WARNING:

In the event of a "serious incident" related to the product that directly or indirectly resulted in, could have resulted in, or may result in any of the following:

- a) Death of the patient, user, or another person, or
- b) Temporary or permanent deterioration of the health of the patient, user, or another person, or
- c) A serious public health threat,

The above "serious incident" must be reported to the manufacturer and the competent authority of the EU Member State where the user or patient resides. In Poland, the responsible authority is the Office for Registration of Medicinal Products, Medical Devices, and Biocidal Products.

WARNING:

If you experience pain, allergic reactions, or any concerning or unclear symptoms related to the use of the medical device, consult a healthcare professional.

WARNING: It is strictly forbidden to use the product in any way other than its intended purpose.

PRECAUTIONS

Failure to follow usage recommendations may result in serious injury.

While using:

Firmly grip the handle.

Keep the elbow crutch as vertical as possible.

The crutch is designed to help the user bear weight, and while using it, always take slightly longer steps.

The maximum static weight the crutch can bear is 120 kg; the user should never exceed this weight.

Use extreme caution when using elbow crutches on carpets, wet, snowy, or any potentially slippery surfaces (leaves, sand, gravel). Do not use crutches on icy surfaces.

In the event of a fall, let go of the crutches – do not hold onto them.

Never hang bags or other objects on the crutches.

Locking mechanisms should not be repeatedly opened and closed unnecessarily, as this may weaken or break the locking button or mechanism.

Never attempt to modify the elbow crutches. They are designed for safety. For the same reason, only use manufacturer-approved replacement parts.

Elbow crutches are solely a walking aid. Never use them for other purposes, such as lifting or leverage.

BEFORE EACH USE, CHECK THAT:

The locking mechanism is properly installed and secured.

The base/tip is in perfect condition. A worn-out tip must be replaced immediately.

The handle is fully inserted.

If any damage is detected, DO NOT use the product. Otherwise, injury may occur. Consult a healthcare professional.

HEIGHT ADJUSTMENT

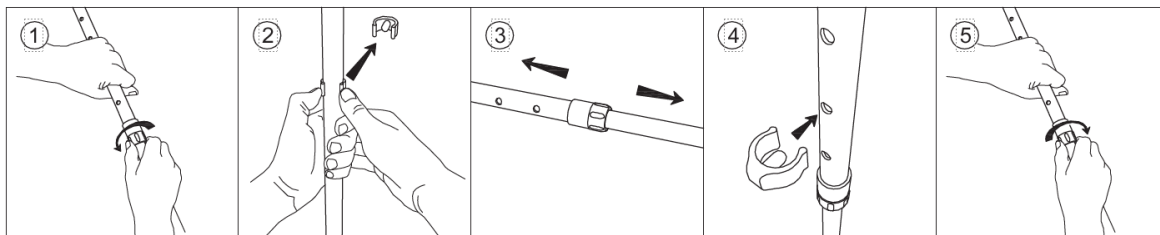
Loosen the **locking nut** by turning it **counterclockwise** (1).

Remove the **clamp with the pin** (2).

Extend the **lower extension** to approximately the desired height (3), and set the exact height by selecting **one of the two holes** in the lower extension.

Align the holes in the lower extension beam and **insert the clamp with the pin** (4).

Secure the set length by **tightening the locking nut** by turning it **clockwise** (5).

Before placing full weight on the adjusted crutch, verify that the locking mechanism is properly secured by gradually applying pressure.

The crutch must be adjusted to the user's height. Incorrect adjustment may reduce the effectiveness of the crutches and require excessive effort from the user. If in doubt, seek advice from your doctor or healthcare professional. If using two elbow crutches, ensure they are adjusted to the same height.

CLEANING

Failure to maintain and clean the crutches may result in serious injury.

Regularly clean the elbow crutches. Wash the handles and tips with clean water. Wipe all surfaces with a damp, soft cloth and allow them to dry before use. Do not use abrasive cleaners, chemicals, greasy products that could make the crutches slippery, or any bleaching agents.

Regularly check all components of the elbow crutch. For safety, a healthcare professional should inspect the crutches every six months.

Regularly examine parts, including hardware, supports, and plastic components for deformation, corrosion, cracks, wear, or compression.

Inspect the following parts for wear or cracks: base (tip), handle, locking mechanism. The base of the crutch should be checked regularly, as it wears out at different rates depending on the surface it is used on.

For safety and continued use, worn or broken parts should be replaced immediately. Use only manufacturer-approved replacement parts.

Spare parts can be obtained from a pharmacist or seller.

The only modification allowed is the replacement of worn-out parts.

If the product has not been used for an extended period, a healthcare professional should inspect it before reuse.

STORAGE

Failure to follow storage conditions may damage the elbow crutches and pose an injury risk.

Do not store the product in places exposed to extreme temperatures (e.g., near a window or heater).

If handles are exposed to extreme temperatures (above 37°C or below 0°C), high humidity, or moisture, check that they do not rotate on the handlebar before use. If they do, damage or injury may occur. Ensure all crutch components are securely fastened before use.

If exposed to extreme temperatures (above 41°C), the crutch may become hot and cause first-degree burns. Allow the crutch to cool before use.

Keep away from fire and flames.

Follow the storage conditions outlined below:

Store in a dry room at a temperature of 5°C to 41°C.

Relative humidity should be between 30% and 70%.

Protect the crutches from dust, corrosion (abrasive materials, sand, seawater, salty air), or greasy environments by storing them in a protective cover.

DISPOSAL

The crutches are made of aluminum tubes and plastic components. These materials are inert, non-toxic, and non-biodegradable. For disposal purposes, the products are considered municipal waste, and local regulations must be followed. Please act responsibly and recycle the product at an appropriate facility when it is no longer usable.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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