en

[REF] TR301

DESCRIPTION OF DEVICE & CONTENTS

Trinity Adhesive Remover

Sting free medical adhesive remover spray (50ml).

INTENDED USE

Removal of adhesives, dressings and appliances from skin.

INSTRUCTION FOR USE

1. Spray around the edge of the adhesive and wait a few seconds.
2. Gently peel adhesive away from the skin using additional spray to ease removal if necessary.

INFORMATION

Dries in a few seconds.
Sprays at any angle.

CAUTION

- Avoid direct use on wound beds.
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.

WARNINGS

Use only as directed. Pressurised container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C. Keep out of reach of children.

EXTREMELY FLAMMABLE

SERIOUS INCIDENT REPORTING

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

SAFE DISPOSAL INSTRUCTIONS

Dispose of product according to national or local laws.

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