## KING SOFT TOILET SEATS


WHITE & BLACK

EN USER MANUAL  
DE BENUTZERHANDBUCH  
FR MANUEL DE L'UTILISATEUR  
ES MANUAL DE USUARIO  
IT MANUALE D'USO  
PT MANUAL DO USUÁRIO  
SL NAVODILA ZA UPORABO  
HR KORISNIČKI PRIRUČNIK  
HU HASZNÁLATI UTASÍTÁS  
RO MANUAL DE UTILIZARE

UK ПОСІБНИК КОРИСТУВАЧА  
SK POUŽIVATEĽSKÁ PRÍRUČKA  
CS UŽIVATELSKÝ MANUÁL  
PL INSTRUKCJA OBSŁUGI  
LT VARTOTOJO VADOVAS  
ET KASUTUSJUHEND  
LV LIETOTĀJA ROKASGRĀMATA  
FI OHJEKIRJA  
SV ANVÄNDARMANUAL  
NL HANDLEIDING

KING-CTS01-KN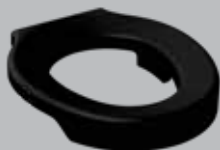

KING-CTS06-KN

|           |                               |       |
|-----------|-------------------------------|-------|
| <b>EN</b> | USER MANUAL .....             | 4-5   |
| <b>DE</b> | BENUTZERHANDBUCH .....        | 6-7   |
| <b>FR</b> | MANUEL DE L'UTILISATEUR ..... | 8-9   |
| <b>ES</b> | MANUAL DE USUARIO .....       | 10-11 |
| <b>IT</b> | MANUALE D'USO .....           | 12-13 |
| <b>PT</b> | MANUAL DO USUÁRIO .....       | 14-15 |
| <b>SL</b> | NAVODILA ZA UPORABO .....     | 16-17 |
| <b>HR</b> | KORISNIČKI PRIRUČNIK .....    | 18-19 |
| <b>HU</b> | HASZNÁLATI UTASÍTÁS .....     | 20-21 |
| <b>RO</b> | MANUAL DE UTILIZARE .....     | 22-23 |
| <b>UK</b> | ПОСІБНИК КОРИСТУВАЧА .....    | 24-25 |
| <b>SK</b> | POUŽÍVATEĽSKÁ PRÍRUČKA .....  | 26-27 |
| <b>CS</b> | UŽIVATELSKÝ MANUÁL .....      | 28-29 |
| <b>PL</b> | INSTRUKCJA OBSŁUGI .....      | 30-31 |
| <b>LT</b> | VARTOTOJO VADOVAS .....       | 31-33 |
| <b>ET</b> | KASUTUSJUHEND .....           | 34-35 |
| <b>LV</b> | LIETOTĀJA ROKASGRĀMATA .....  | 36-37 |
| <b>FI</b> | OHJEKIRJA .....               | 38-39 |
| <b>SV</b> | ANVÄNDARMANUAL .....          | 40-41 |
| <b>NL</b> | HANDLEIDING .....             | 42-43 |

# KING SOFT TOILET SEATS

WHITE & BLACK

---

## INTENDED USAGE:

The *Soft Cushions for Toilet Seat KING* are designed to make the toilet seat more comfortable, soft, and cosy. The soft cushion wraps the toilet seat, partially protecting it from dirt and making it easier to keep it clean. In addition, the wrinkled surface provides more friction, reducing the slip risk for the user. The soft cushion can be easily applied and removed from the toilet seat.

## INDICATIONS AND CONTRAINDICATIONS:

The soft cushion is aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who may benefit from sitting on a soft and warm to the touch seat while using the toilet. During sitting, the tender material of the soft cushion deforms, ensuring an increase of the user support surface and allowing for better distribution of the body weight. This makes this accessory particularly suitable if the user suffers from bedsores or diseases which could cause it, such as diabetes. The ability to sit independently is required during use. Using the soft cushion in case of perception disorders or balance problems while sitting is not recommended.

## SAFETY WARNING AND INDICATIONS:

- Before applying the soft cushion, be sure the toilet seat is clean and dry.
- Before every use, be sure the soft cushion correctly wraps the toilet seat.
- During the assembly, strictly adhere to what is shown in the User Manual.
- Do not modify the soft cushion in any way.
- Do not use the soft cushion if damaged or altered in any way.
- During use, avoid positions that keep the body weight too much frontward.
- The soft cushion must not be stored outdoors.

## MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

**The soft cushion must be cleaned strictly by hand.**

## REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

## CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

## INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at [info@kingbathrooms.com](mailto:info@kingbathrooms.com) and the responsible authority in your country.

| ITEM CODE            | DESCRIPTION                             | PAGE      |
|----------------------|-----------------------------------------|-----------|
| <b>KING-CTS01-KN</b> | Soft cushion for toilet seat white King | <b>44</b> |
| <b>KING-CTS06-KN</b> | Soft cushion for toilet seat black King | <b>44</b> |

| ICON                                                                             | DESCRIPTION         |
|----------------------------------------------------------------------------------|---------------------|
|  | Made in Italy       |
|  | Tools Free Assembly |
|  | 2 Years Warranty    |

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**LM Global Design Ltd.**