ESENTA™

Stomahesive®

Powder / Пудра / Puder / Pudr / Pulver / Tapadópor / Puderis / Pudra / Пудра / Puder /
Pudră / Puder / Zässp / Posip

ENGLISH

Intended use: Intended to absorb moisture and protect skin from damage, and to facilitate the application of ostomy systems.

Indications for use: For use with a colostomy, ileostomy or urostomy.

Caution:

Do not use with patients with known sensitivity to Stomahesive® Powder or its constituents.

Directions for use:

- Wash hands and dry well.
- Apply the powder as a thin layer as often as necessary. Gently puff the powder onto the moist area. If any of the powder lands on dry skin gently wipe away with a damp cloth. If the area is very moist you may have to repeat this process once or twice. Once completed it is safe to apply the pouch directly. Do not apply the powder to the baseplate/wafer.
- Keep cap tightly closed.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

Waste Disposal

Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

Precautions and warnings:

Do not apply the adhesive powder directly under the ostomy system support as this may lead to a delay in adherence.

© 2025 Convatec.

™/® indicates a trademark of the Convatec Group.

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Convatec Limited
First Avenue, Deeside Industrial Park
Deeside, Flintshire, CH5 2NU, UK

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Effective

ARTWORK SPECIFICATION	
Artcode	SI2454
Artwork Description	INS ESENTA S/HESIVE PDR 25G CEE
ICC*	025535
SAP*	1739974
KOM/REF*	N/A
IS Code*	N/A
Component Type	Insert
LR#	LR-004970
Existing Art Code being Revised	N/A
REQUIREMENT	SPECIFICATION
Bucket	CEE
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 180 mm x 252 mm, Folded size: 90 mm x 31.5 mm
Color(s) / Colour(s)	Black
Specification Reference(s)	TF13-040
Bar Code: UPC (A) or GTIN**	N/A
Bar Code: PZN – Code 39**	N/A
Bar Code: JAN-13**	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade “C” or better.	
HISTORY	
12-Apr-2024 / SI2454: New artwork created. 19-Sep-2024 / SI2454: Added Keep dry and EC REP symbol with explanations. 11-Nov-2024 / SI2454: Deleted UKCA symbol.	

WI-0010, Ver. 7.0