Instructions for Use

Cutimed® Hydro L / Cutimed® Hydro B**Hydrocolloid dressing****Product description**

Cutimed® Hydro is a range of sterile hydrocolloid dressings designed to maintain a moist wound environment. The dressing material interacts with the wound exudate to form a soft gel.

Cutimed® Hydro L is a flexible dressing. It is comfortable and has a top layer of low friction film, which allows close and easy monitoring of the healing process without dressing removal. It can be used for the management of dry to low exuding wounds. Whereas Cutimed® Hydro B is a dressing with a tapered edge and a thicker core. Compared with Cutimed® Hydro L, Cutimed® Hydro B has stronger liquid absorption capacity. The edge part is thinner than the core part, so it can be bonded to the skin for a long time. It can be used for management of dry to moderately exuding wounds.

Cutimed® Hydro is made of three parts: hydrocolloid, polyurethane film and release paper. The hydrocolloid layer is composed of sodium carboxymethyl cellulose (CMC) and hot melt adhesive. It has the function of absorbing liquid and allows the dressing to adhere stably to the wound and surrounding skin. The polyurethane film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria. This product can provide a moist environment for the wound.

Function principle

In the presence of wound exudate, the hydrocolloid absorbs liquid and forms a gel that does not adhere to the wound. This gel forms a moist environment at the wound interface, supports the wound healing process and reduces the risk of maceration of the surrounding skin. Most of the gel is removed together with the dressing, resulting in little or no damage to the newly formed tissue. The polyurethane film is moisture vapor permeable, waterproof, and impermeable to liquids, bacteria, which helps to provide a moist healing environment for the wound.

Intended purpose & indications**Cutimed® Hydro L**

Cutimed® Hydro L is a wound dressing for dry to low exuding wounds that protects the wound from foreign contamination, absorbs small amounts of exudate and provides a moist healing environment for the wound.

Indications:

Dry to low exuding wounds including donor sites, postoperative wounds, skin abrasions.

Cutimed® Hydro B

Cutimed® Hydro B is a wound dressing for dry to moderately exuding wounds that protects the wound from foreign contamination, absorbs exudate and provides a moist healing environment for the wound.

Indications:

Dry to moderately exuding wounds including pressure ulcers, leg ulcers, donor sites, postoperative wounds, skin abrasions.

Intended user

The product shall be used by or under the supervision of a healthcare professional. Please contact your healthcare professional if you are unsure whether the product is appropriate for you.

Intended patient population**Cutimed® Hydro L**

Patient who has the indications: Dry to low exuding wounds including donor sites, postoperative wounds, skin abrasions.

Cutimed® Hydro B

Patient who has the indications: Dry to moderately exuding wounds including pressure ulcers, leg ulcers, donor sites, postoperative wounds, skin abrasions.

Clinical benefit

Relieve patient pain during treatment.

Contraindications

- Do not use on highly exuding wounds.
- Do not use on the patients with a known hypersensitivity to the product itself or to its components.

Complications or side-effects

Very occasionally, there will be some potential risks associated with the use of Cutimed® Hydro L and Cutimed® Hydro B dressing with the following:

- adverse reactions (intolerance, erythema, hyper granulation etc.)
- wound infections
- risk of maceration (fluid leakage)

Directions for use

Firstly, ensure the product is kept clean and sterile before using. Do not use if package is damaged. Clean the wound area in accordance with normal procedure. Ensure the peri-wound skin is dry.

Application:

- Select a size of hydrocolloid dressing large enough to overlap the wound edges by at least 2–3 cm. If necessary, cut to shape (A).
- Remove the dressing from the pouch, partially tear off the release film and apply the adhesive side to the wound (B).
- Remove the remaining liner and smooth dressing down onto skin (C).
- For dressing removal, press down on the skin outside of the dressing and gently lift an edge of the dressing. Gently loosen the edges until all are free from the skin surface.

Note 1:

The dressing change interval may be several days. Change the dressing before it is fully saturated, or as indicated by clinical practice. The same product cannot be used for more than 7 days; continuous use cannot be more than 30 days.

Note 2:

Frequent dressing changes on wounds with damaged or delicate skin surrounding the wound is not recommended.

Precautions/Warnings

- Do not reuse. Reusing will cause cross-contamination.
- Do not use if package is damaged or open.

Storage and transport conditions

The product should be stored at 15–25°C and kept away from direct sunlight. The transportation should prevent heavy weight, sun and rain.

Disposal

Disposal should be handled according to local environment procedures.

Other information

- The intended user does not need training for this product.
- If any serious incident has occurred in relation to the use of this dressing, it should be reported to Winner Medical Co., Ltd. and the competent authority of your state.

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