EN - Instruction for use

Product: **clevesorb** – super absorbent dressing

We recommend reading the instructions for use carefully always before using a package of clevecare, as the information is updated regularly.

Intended use:

clevesorb - super absorbent dressing is adaptable to different phases of wound especially to all kinds of highly exudated wounds and all kinds of post-operation surgery wound and injuries, such as burn wound, ulcer, pressure ulcer, refractory chronic wounds.

Contraindications:

We recommend that **clevesorb** should not be used on dry or covered with a hard black necrotic wound.

Side-effects:

Not identified.

Warning and precautions:

- This product is disposable. Do not use if the inner package is damaged.
- This product is sterilized. Use immediately upon opening to maintain sterility.
- This product shall not be reused.
- The products have a shelf-life of 3 years. Do not use beyond the expiration date.
- For optimal performance, retain a single piece of the product for no longer than 7 days.
- Change the dressing promptly when it swells and approaches the edges after absorbing exudate.
- Store the product in a cool, dry place, protected from light, and ensure adequate ventilation.

Contents:

The instructions for use of the **clevesorb** are included. Please note that each step of the instructions must be followed in strict chronological order, even if mentioned within the same sentence.

Instructions For Use:

- Cleanse the wound in accordance with local clinical protocols.
- Choose a clevesorb dressing size large enough to extend 3cm beyond the wound edges onto dry, healthy peri wound skin.
- Remove clevesorb dressing from the packaging using a clean technique.
- Place the white side of the dressing directly onto the wound and secure it with a secondary dressing, tape, or bandage (ensure the blue side faces outward—"Blue side to the sky").
- Use a retention bandage to ensure the dressing remains securely in place on the wound.

Removal:

Careful and correct clevesorb dressing removal is vital to minimize the risk of damaging the wound bed and new granulating tissue.

- Remove the retention dressing or bandage and carefully lift the clevesorb dressing away from the wound.
- The dressing should be removed slowly and carefully.
- As the dressing is removed, the newly exposed skin should be supported.
- Soiled dressings should be discarded in the appropriate clinical waste receptacle.

Frequency of Dressing Change:

- The use of clevesorb dressings should always be re-evaluated if wound exudate levels change.
- During the early stages of wound management, observe the wound daily for signs of infection or other complications (or as recommended by local clinical protocols).
- Infected wounds should be inspected and treated as per local clinical protocol.
- The frequency of dressing change is determined by the condition of the wound and local clinical protocols.

Shelf-life and storage information:

Shelf-life: The products have a shelf-life of 3 years. After expiry of the expiry date, the products must no longer be used.

Storage information: The product should be stored in a horizontal position. Keep dry. Keep away from sunlight.

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Zhende Medical Co., Ltd. Gaobu Town, 312035 Shaoxing, Zhejiang PEOPLE'S REPUBLIC OF CHINA