 Fidia Farmaceutici S.p.A. Via Ponte della Fabbrica 3/A 35031 Abano Terme (PD) - Italy		
Under no circumstances should this artwork be altered without prior permission from Fidia Farmaceutici www.fidiapharma.com	Versione/Version:	1
	Data Rev.: Rev. date:	05.03.2025
Prodotto/Product name: IST HYALO4 CARE START EAST EU		
Codice/Item: 33001971	Revisione/Revision: A	
Data/Date: 05.03.2025		
Disegno No./Master No.: IST 030		
Fustella/Die Cut: N/A		
Dimensioni/Size: (in mm) 139 x 380		
Tecnica di Stampa/Print Process: N/A		
Corpo del testo/Body text: 6 pt		
Testo più piccolo/Smallest text: 6 pt		
Interlinea/Leading: 6,5 pt		
Scala Orizzontale/Horizontal Scale: 100%		
Braille: No	Pharmacode No: N/A	
Edge Code: N/A		
Motivo della modifica/Change Description: New artwork		
Colori/Colors No: 1		Artwork Operator
P 2955 C		Francesco Cappellari A.O Signature
Vernici/Varnish No: N/A		

Orig. IND • Version 3.1/01.2024

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The design and the text are Fidia S.p.A. or a Company member of Fidia S.p.A. property

START

EN | HYALO⁴ CARE START INSTRUCTIONS FOR USE

Description of the product
HYALO⁴ CARE START is a fluid ointment for topical use containing sodium hyaluronate obtained by bacterial fermentation, as the principal component, and bacterial collagenase obtained from a non -pathogenic *Vibrio alginolyticus* strain, as an incorporated medicinal substance.
HYALO⁴ CARE START allows the preparation of the wound bed, thanks to the hydrating properties of hyaluronic acid that promotes the natural repair process and protects the new viable tissue. In fact the process of cicatrization of the lesions takes place more rapidly in a moist environment. The bacterial collagenase assists in the wound bed preparation.

Composition
Principal component: sodium hyaluronate 0.2%
Other components: collagenase (not less than 2.0 nkat/g of ointment), light liquid paraffin, white petrolatum.

Indications
Local management of chronic ulcers (i.e. pressure sores, vascular ulcers of the legs, diabetic ulcers). In particular, it is intended to provide an optimal moist environment and wound bed preparation that supports the natural healing process.

Dosage and mode of administration
The wounds should be cleaned and disinfected before the application. A sufficient humidity in the area of the lesion must be present to assure a successful treatment. Dry lesions must be moistened with physiological solution (0.9% NaCl) or with solutions well tolerated by the tissue, as the glucose solution. Hard eschars should be softened by applying a wet dressing. Apply a layer of about 2 mm of HYALO⁴ CARE START on the wound once a day or as per your doctor's advice. Cover the treated area with a sterile dressing to assure an appropriate contact.

Pregnancy and Lactation
HYALO⁴ CARE START can be used during pregnancy and lactation, if indicated by physician.

Contraindications
Patients with ascertained individual hypersensitivity to the product components must not use HYALO⁴ CARE START.

Warnings and precautions
It is suggested not to use the product for a period longer than 30 days. Consult your physician in case of infection complication. Tetracycline should not be used locally with HYALO⁴ CARE START.
Do not use the product concomitantly with soaps/detergents, disinfectants containing quaternary ammonium salts, silver and other heavy metals, which can reduce the action of the product. Ask your physician the possible products to be used.
Do not use on exposed cartilage or tendons.
Avoid contact with eyes and mucosae.
In case of accidental ingestion or inhalation consult the physician immediately. Avoid the direct contact of the container with the wound.
After each application put the cap back on the container.
Do not use in case of package damage.
The product can be used up to 36 months after the first opening of the tube.
Do not use the product after the expiry date reported on the package.
The expiry date refers to the product stored correctly.
After use, dispose according to applicable national regulations
Keep out of sight and reach of children.

Undesiderable effects
Undesiderable local effects such as pain, itching, burning and erythema may occur. Undesirable effects of systemic type cannot be excluded in patients with ascertained hypersensitivity towards the components of the product.

Storage
Store at temperature not exceeding 30°C.
Do not freeze.

Presentation
Tube of 30 g.
Date of the latest revision of the instructions for use: May 2020

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