AT53103/AT53104/AT53105/AT53106
INSTRUKCJA UŻYTKOWNIA, ORTEZA USZTYWNIAJĄCA PALCA
INSTRUCTION MANUAL, FINGER SPLINT ORTHOSIS
GEBRAUCHSANWEISUNG, FINGERORTHOSE
NÁVOD K POUŽITÍ, ORTÉZA NA ZTUHNUTÍ PRSTU
NÁVOD NA POUŽITIE, ORTÉZA NA SPEVNENIE PRSTA
NOTICE D'UTILISATION, ORTHÈSE IMMOBILISANTE POUR DOIGT
GEBRUIKSAANWIJZING, VINGERSPALK ORTHOSE
MANUAL DE INSTRUCCIONES, ÓRTESIS INMOVILIZADORA DE DEDO
MANUALE D'USO, ORTESI PER DITO IMMOBILIZZANTE
BRUKSANVISNING, FINGERORTOS

PL

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Thank you for purchasing our product. Please carefully read this user manual before use. Do not use the product before becoming familiar with this manual and understanding it. If you do not understand the warnings, notes, and recommendations, consult a healthcare professional or the seller to avoid damage to the product or bodily injury.

WARNING: Check all parts for damage caused during transport. If such damage is observed, DO NOT use the product. For more information, contact the seller.

TARGET PATIENT GROUP

Individuals suffering from diseases, dysfunctions, or injuries for which this product is intended (see the section on product purpose in this manual). The product may be purchased directly by the user or based on the recommendation of a doctor, therapist, or other specialist. In either case, available sizes, required functions, product variants, indications and contraindications, as well as manufacturer-provided information must be considered.

APPLICATION/INDICATIONS

Finger splints are orthopedic devices designed to immobilize injured or weakened finger joints.

Indications for use include:

Phalangeal fractures (e.g., mallet finger), Finger tendon injuries, Sprains, bruises, dislocations, Postoperative recovery, Degenerative and inflammatory joint diseases (e.g., RA), Correction of interphalangeal joint deformities, Temporary joint protection during healing Or rehabilitation

Applications: Immobilization of the injured finger joint, Reduction of pain, inflammation, and risk of further injury, Stabilization during the healing and recovery process, Protection of the joint during daily activities or Steep, Post-traumatic and postoperative rehabilitation

CONTRAINDICATIONS

Physical or mental limitations (e.g., vision impairment) preventing safe product use

Circulatory disorders in the hand or Finders, Open wounds, ulcers, or skin infections at the contact area, Hypersensitivity or allergy to any of the materials, Permanent deformities preventing proper fitting, Numbness, cyanosis, or pain while wearing (requires consultation),

Inability to self-monitor skin condition (e.g., neuropathy)

FEATURES

Lightweight construction – does not burden the finger, allows all-day wear

Breathable materials – reduce irritation and sweating

Rigid or flexible structure – depending on type (stack, buddy, aluminum)

Adjustable – elastic straps or Velcro bands allow fitting to various finger sizes

Latex-free and hypoallergenic finish – suitable for sensitive skin or allergies

Easy application – can be put on and taken off with one hand

SIZES

AT53103

S: 8 cm × 3 cm M: 10 cm × 3 cm L: 12 cm × 3 cm

AT53104

S: 8 cm × 7 cm M: 8 cm × 8 cm L: 8 cm × 9 cm

AT53105

S: 8 cm × 2.5 cm M: 10.5 cm × 2.5 cm L: 12.5 cm × 2.5 cm

AT53106

S: 4 cm × 3 cm M: 7 cm × 3 cm L: 9.5 cm × 3 cm

COMPONENTS LIST: Finger splint orthosis, user manual

USAGE

Ensure the finger is clean and dry: Wash and dry the hand before use to prevent skin irritation and moisture buildup.

Position the splint along the finger: Adjust the splint length to stabilize the joint or injured area. For stack splints, the fingertip should rest against the splint.

Secure with straps or tape: Wrap straps gently but firmly around the finger. Ensure the splint is not too tight – it should not restrict circulation or cause discomfort.

Check fit and comfort: The finger should be immobilized but not numb. Adjust straps if necessary.

Use as directed by your doctor or therapist: Duration of wear and immobilization type should be individually prescribed. If pain, numbness, or swelling occurs – remove the splint and consult a doctor.

Daily skin check: Remove the splint briefly each day to inspect the skin for redness, abrasions, or pressure marks.

WARNING:

If a "serious incident" occurs related to the product that has led, could have led, or may lead to any of the following:

- a) death of the patient, user, or another person,
- b) temporary or permanent deterioration of health, or
- c) serious threat to public health,

this "serious incident" must be reported to the manufacturer and the relevant competent authority of the Member State where the patient or user resides. **WARNING:** If pain, allergic reactions, or other concerning symptoms occur during use, consult a healthcare professional.

WARNING: The product must not be used in any way other than its intended purpose.

CLEANING

Hand wash: Use mild soap and lukewarm water. Gently wipe the splint with a soft cloth or sponge.

Disinfection (if needed): Wipe the splint surface with 70% isopropyl alcohol or disinfectant wipes. Do not soak the product.

Drying: Air-dry completely before reuse. Do not expose to direct heat (e.g., hair dryer) to avoid deformation.

STORAGE

Store in a dry place, away from direct sunlight.

DISPOSAL

Once out of use, dispose of the product as household waste. Follow waste segregation regulations. Care for the environment.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiślańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiślańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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