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Produktdescription

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IN Instructions for use

Produkt description RespOsr Super is a sterile superabsorbent dressing used on superficial, moderately to highly exuding wounds. The absorbent core absorbs and binds exudate.

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Properties and mode of action

RespOsr Super rapidly absorbs exudate and binds it within the absorbent core. Exudate removal eliminates inhibitory factors from the wound, e.g. proteases. The increased absorbent capacity of RespOsr Super reduces the required frequency of dressing changes. This promotes wound rest and provides additional protection against contamination. Apart from its absorbent quality, RespOsr Super also has a padding effect.

Intended purpose

Single use sterile superabsorbent dressing for long-term treatment of injured skin, acute and chronic, with moderate to high levels of exudate. It is used on adults only, by healthcare professionals in clinical or domestic environments. It can be combined with local antiseptics, primary and secondary dressings as well as under compression bandages.

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Mode of application

- Select RespOsr Super to match the wound size so that the dressing extends at least 1–2 centimetres beyond the wound margins.
- Place the white side of RespOsr Super on the wound so that the green special nonwoven layer faces away from the wound.

- Secure the dressing e.g. with adhesive tape, conforming bandages or, if necessary, compression bandages.
- One dressing can remain on the wound for up to 7 days, depending on the condition of the wound. Change the dressing if clinically indicated or when exudate reaches the edges of the dressing or is noticeable through the green nonwoven top layer.
- In case of infected wounds, a combination with e.g. Atramaun Ag is possible.
- With decreasing amounts of exudate, the use of a suitable hydroactive wound dressing (e.g. HydroTac) is recommended.

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Initial reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Product disposal

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards. Dispose of the medical device with household waste.

Date of revision of the text: 2025-02-03 AU – PAUL HARTMANN Pty Ltd., Macquarie Park, NSW 2113 ZA – HARTMANN Ltd. - Heywood/Greater Manchester OL10 2TT

DE – PAUL HARTMANN AG - 89522 HEIDENHEIM, GERMANY AT – PAUL HARTMANN Ges.m.b.H., 2355 Wiener Neudorf, CH – IW HARTMANN AG - 8212 Neuchâuen

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Contraindications

Do not use RespOsr Super on dry wounds or on exposed bones, muscles or tendons. Do not use RespOsr Super in case of hypersensitivity to any of its components.

Special precautions

- Do not cut the dressing.
- Medical assessment of the wound condition and the causes of wound-healing impairment is necessary before treating wounds with an impaired healing tendency.

Treatment with RespOsr Super cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.

- In all cases, follow the established clinical protocol.
- In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information for use should be read.

Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

EN Instructions for use

RespOsr Super is a combined superabsorbent dressing which consists of four layers of different materials. On the wound side, the product features a soft, white, hydrophilic nonwoven. The inner dressing core consists of soft cellulose fluff blended with liquid-retaining polyacrylate polymers. This absorbent core is enclosed in a thin nonwoven fabric that evenly distributes the liquid. On the side facing away from the wound, the product features a green layer of polypropylene nonwoven, which is water-repellent but permeable to air and allows for gas exchange.

Properties and mode of action RespOsr Super rapidly absorbs exudate and binds it within the absorbent core. Exudate removal eliminates inhibitory factors from the wound, e.g. proteases. The increased absorbent capacity of RespOsr Super reduces the required frequency of dressing changes. This promotes wound rest and provides additional protection against contamination. Apart from its absorbent quality, RespOsr Super also has a padding effect.

Intended purpose Single use sterile superabsorbent dressing for long-term treatment of injured skin, acute and chronic, with moderate to high levels of exudate. It is used on adults only, by healthcare professionals in clinical or domestic environments. It can be combined with local antiseptics, primary and secondary dressings as well as under compression bandages.

Indications RespOsr Super is suitable for the treatment of superficial, moderately to highly exuding wounds: acute wounds (traumatic wounds, post-operative wounds, lymphatic wounds) and chronic wounds (decubitus/pressure sores, venous, arterial or mixed leg ulcers, tumour wounds). RespOsr Super can be used under compression bandages.

Mode of application Select RespOsr Super to match the wound size so that the dressing extends at least 1–2 centimetres beyond the wound margins. Place the white side of RespOsr Super on the wound so that the green special nonwoven layer faces away from the wound. Secure the dressing e.g. with adhesive tape, conforming bandages or, if necessary, compression bandages. One dressing can remain on the wound for up to 7 days, depending on the condition of the wound. Change the dressing if clinically indicated or when exudate reaches the edges of the dressing or is noticeable through the green nonwoven top layer. In case of infected wounds, a combination with e.g. Atramaun Ag is possible. With decreasing amounts of exudate, the use of a suitable hydroactive wound dressing (e.g. HydroTac) is recommended.

Contraindications Do not use RespOsr Super on dry wounds or on exposed bones, muscles or tendons. Do not use RespOsr Super in case of hypersensitivity to any of its components.

Special precautions Do not cut the dressing. Medical assessment of the wound condition and the causes of wound-healing impairment is necessary before treating wounds with an impaired healing tendency. Treatment with RespOsr Super cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used. In all cases, follow the established clinical protocol. In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and following a clinician's recommendation.

Product disposal In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards. Dispose of the medical device with household waste.

Date of revision of the text: 2025-02-03 AU – PAUL HARTMANN Pty Ltd., Macquarie Park, NSW 2113 ZA – HARTMANN Ltd. - Heywood/Greater Manchester OL10 2TT DE – PAUL HARTMANN AG - 89522 HEIDENHEIM, GERMANY AT – PAUL HARTMANN Ges.m.b.H., 2355 Wiener Neudorf, CH – IW HARTMANN AG - 8212 Neuchâuen

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