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EN Instructions for use

Product description and composition

HydroClean Mini is a hydroactive wound dressing that contains, as a core component, a superabsorbent polyacrylate (SAP) embedded in cellulose fibres and activated (premoistened) with Ringer's solution. The wound pad is covered with a knitted fabric consisting of polypropylene.

Properties and mode of action

HydroClean Mini delivers Ringer's solution to the wound for up to three days. During this time, interactive and continuous wound irrigation takes place, due to the wound dressing pad also absorbing wound exudate. The SAP used in HydroClean Mini inactivates matrix metalloproteinases (MMP) which may impair wound healing.

Intended use

HydroClean Mini is suitable for wet/moist treatment of wounds. The dressing should be used by healthcare professionals or laypersons trained by healthcare professionals.

Indications

HydroClean Mini is indicated for the wet management of dry, light and moderately exuding wounds, in particular of wounds with impaired healing tendency. The product is especially suitable for the treatment of chronic and poorly healing wounds during the cleansing and granulation stage.

Mode of application and dressing change

The packaging and the dressing should be checked for damage. HydroClean Mini should not be used if damage is detected. Before opening the product packaging, inspect it carefully for signs of damage. In particular, examine the integrity of the packaging surfaces and the entire seal area completely under suitable lighting conditions (white light or daylight). Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals. The dressing change should be carried out according to current medical standards. It should especially be carried out under aseptic conditions. The dressing must therefore not be touched with unsterile hands/gloves or instruments. The wound pad should not be pressed out prior to its application.

For deep wounds:

- HydroClean Mini is placed into the wound.
- HydroClean Mini must be in contact with the wound bed.
- HydroClean Mini should be covered with a HydroClean wound dressing pad.
- If ault veiller à choisir un pansement suffisamment grand pour assurer une fixation sûre.
- HydroClean Mini should be secured with suitable dressing retention material.
- If HydroClean Mini is inserted into a wound with a condition that does not allow it to be covered with a HydroClean dressing, a film dressing can be used instead.

For less deep wounds:

- HydroClean Mini is placed onto the wound.
- HydroClean Mini must be in contact with the wound bed.
- HydroClean Mini should overlap the wound edges.
- HydroClean Mini should be secured with a film dressing.
- Attention should be paid to choosing a large enough film dressing to ensure secure fixation. This applies in particular to localisations where the fixation is stressed considerably by movement. When using a film dressing one should also ensure that Ringer's solution does not leak from the wound dressing pad as a result of pressure, thus preventing secure fixation.

HydroClean Mini is also suitable to be used under compression bandages or stockings. The dressing can remain in situ for up to three days, depending on clinical judgement. The dressing should be changed if the absorption capacity is exhausted or the dressing dries out prematurely. In most cases, HydroClean Mini can be removed from the wound without causing additional pain. Should the wound dressing pad adhere to the wound, it can be irrigated with a sterile physiological saline or Ringer's solution prior to removal to loosen the adhesion.

Special precautions

Before treating wounds with impaired healing tendency, the wound condition and the causes of the impairment must always be assessed by a physician or other healthcare professional. Treatment with HydroClean Mini cannot replace treatment of the cause of impaired wound healing. HydroClean Mini must not be cut to size or otherwise damaged mechanically. If HydroClean Mini is applied on the sole of a foot, the patient must not stand or walk on it because the dressing may burst. If the dressing is damaged, SAP particles and fibers may get into the wound. They should be rinsed out with a sterile physiological saline or Ringer's solution. Infected wounds and wounds highly exposed to bacteria should only be treated under the supervision of a physician or other healthcare professional, particularly if a film dressing is used for fixation. Moisturizing dressings like HydroClean Mini are not recommended to be used to treat dry necrotic foot diseases caused by arterial occlusive disease. Third degree burns must be treated by surgical procedures prior to the use of HydroClean Mini. Topical medication or disinfectants should not be applied during treatment with HydroClean Mini. The dressing may absorb or otherwise compromise the efficacy of the agents. HydroClean Mini should not be used to treat heavily exuding wounds. If the skin around the wound is particularly sensitive and impaired, it is advisable to protect the skin with suitable skin protection. In patients with a risk of excessive bleeding (e.g. due to blood-thinning medications), HydroClean Mini should be removed with caution at each dressing change. If necessary, HydroClean Mini should first be irrigated with Ringer's solution or physiological saline solution as described above. Safety and effectiveness in pregnant women and children have not been established.

Kontraindications

Do not use HydroClean Mini on patients with an intolerance to any of its components.

Side effects

At the start of the treatment with HydroClean Mini, breakdown of irreversibly damaged tissue can lead to an increase in the size of the wound. This can be a sign that wound healing has commenced. Reddening of the wound edges can occur during wet/moist treatment; this is normally a sign that blood circulation has been reactivated as the wound begins to heal. However, the cause must be assessed individually in each case. HydroClean Mini therapy itself does not cause any pain. However, an open wound is often highly sensitive to mechanical stimuli. The question whether an adequate analgesic therapy is indicated must be considered on a case-by-case basis. Whilst wet/moist treatment with Ringer's solution does not lead to maceration of vital cells, it can cause swelling of the soaked plantar horn layer, which then appears as a whitish coating. In rare cases, unspecific activation of gluteal and leg muscles had been observed in paraplegic patients with sacral wounds. The symptoms were immediately resolved after the removal of the dressing.

Product disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Special instructions

Re-use of HydroClean Mini may cause skin irritations, a prolonged healing time, pain, and/or infections. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance.

Incident reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Date of revision of the text: 2022-11-17
FR – Laboratoires PAUL HARTMANN S.à.r.l. - Châtenois – 67607 Sélestat CEDEX

AT – PAUL HARTMANN Pty. Ltd. - Marquarie Park, NSW 2133
GB – PAUL HARTMANN Ltd. - Heywood/Greater Manchester OL10 2TT
ZA – HARTMANN South Africa - Northridge, Johannesburg, Gauteng

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IT – PAUL HARTMANN S.p.A. - 37139 Verona