 Fidia Farmaceutici S.p.A. Via Ponte della Fabbrica 3/A 35031 Abano Terme (PD) - Italy		
Under no circumstances should this artwork be altered without prior permission from Fidia Farmaceutici www.fidiapharma.com	Versione/Version:	3
	Data Rev.: Rev. date:	05.05.2025
Prodotto/Product name: Istr. Hyalo4 Care Cream East EU1		
Codice/Item: 33001965	Revisione/Revision: A	
Data/Date: 03.03.2025		
Disegno No./Master No.: IST 030		
Fustella/Die Cut: N/A		
Dimensioni/Size: (in mm) 139 x 380		
Tecnica di Stampa/Print Process: Offset		
Corpo del testo/Body text: 6 pt		
Testo più piccolo/Smallest text: 6 pt		
Interlinea/Leading: 7 pt		
Scala Orizzontale/Horizontal Scale: 95%		
Braille: No	Pharmacode No: N/A	
Edge Code: N/A		
Motivo della modifica/Change Description: New artwork		
Colori/Colors No: 1		Artwork Operator
P 2955 C		Alessandro Marchi A.O Signature
Vernici/Varnish No: N/A		D.A.R Signature for approval

Orig./ND • Version 3.1/01.2024

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The design and the text are Fidia S.p.A. or a Company member of Fidia S.p.A. property

Signature for approval	
Legal	
Marketing	
D.A.R.	

EN | HYALO4 CARE CREAM

Product description
HYALO4 CARE CREAM is a topical preparation constituted by hyaluronic acid as its principal component. Hyaluronic acid is a biological polysaccharide (glycosaminoglycan) distributed in the extracellular matrix of most tissues. Due to its hydrophilic properties, it provides hydrated space around and between cells, thus facilitating the migration of cells.

Composition
Principal component: Hyaluronic acid sodium salt at 0.2%
Other components: Polyethylenglycol 400 monostearate, Decyl ester of oleic acid, Emulsifying wax, Glycerol, Sorbitol solution 70%, Sodium dehydroacetate, Methyl p-hydroxybenzoate, Propyl p-hydroxybenzoate, Fragrance, Lactic Acid, Purified water.

Indications
HYALO4 CARE CREAM is indicated for the management of cutaneous irritations and lesions. In particular, it is intended to cover acute wounds (donor sites and post-operative incisions, first and second degree burns) and chronic wounds (vascular and metabolic ulcers and pressure sores).

Dosage and Administration
The wounds or ulcers should be cleaned and disinfected prior to treatment. Apply a thin layer of cream onto the wound surface, two or three times a day for not more than 30 consecutive days. The treated area should be covered with a sterile gauze-pad and a sterile bandage.

Contraindications
Do not administer to patients with ascertained individual hypersensitivity to the product components.

Warnings and Precautions
For external use only.
Do not use in case of package damage.
Do not use the product after the expiry date reported on the package.
The expiry date refers to the product kept in its original package at a temperature below 30 °C.
Discard after 12 months from the first opening.
Avoid direct contact of the container with the wound.
Each tube of HYALO4 CARE CREAM should be used by one patient only in order to reduce the risk of cross infection.
In case of pregnancy, breastfeeding and use in children, consult your doctor before using the product, since the safety and efficacy of HYALO4 CARE CREAM in those cases have not been established.
After use, dispose according to applicable national practice.
Keep out of reach of children.

Adverse reactions
The following local adverse reactions may occasionally occur: application site rash, itching, burning sensation, skin irritation, dryness, erythema, dermatitis, skin discoloration. A transient effect of metallic taste (dysgeusia) may occur with very low incidence and could be related to pre-existing medical conditions of the patients. The symptoms are usually transient, of mild severity and rarely require early termination of treatment.

Interactions
Do not use concomitantly with disinfectants containing quaternary ammonium salts because hyaluronic acid can precipitate in their presence. The concomitant topical treatment of wounds with antibiotics or other local agents has never given rise to interactions or incompatibilities with HYALO4 CARE CREAM.

Storage
Store below 30 °C.

Presentation
Cardboard box containing one 15 g tube
Cardboard box containing one 25 g tube
Cardboard box containing one 30 g tube
Cardboard box containing one 100 g tube
Date of the latest revision of the instructions for use: November 2020

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