REF	Width	Length	Height	Seat Width	Seat Height	Max. User Weight
091357193	54cm	60cm	88.5cm	46cm	48.5cm	160kg
091357201	54cm	60cm	88.5cm	46cm	48.5cm	160kg
091438605	54cm	60cm	88.5–99.5cm	46cm	48.5–60cm	160kg

Operation

This range of mobile shower chairs are fitted with four 10 cm (4") brake castors. Each castor can be braked by pushing down the lever on the castor. This prevents the wheels from rotating or swivelling. To release, the brake the lever should be pushed upward.

The arms on model 09 135 7201 can be removed for sideways transfer. The user must ensure that the arms are fully located before putting any weight on them.

The 09 143 8605 can be adjusted in height. To set the height, pull out the snap locks and move legs to required height. Insert snap locks, ensuring that all 4 legs are at the same height. (Do not bend the plastic legs of the snap locks as this will reduce their grip on the leg and make the leg insecure).

Maintenance

These range of mobile shower chairs are designed for minimum maintenance and easy, modular replacement of parts for long life. It is manufactured from powder coated steel.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. The shower chair may be placed under running water at low pressure or showered down to assist cleaning. Do not use harsh abrasives. To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Daily, before use, check the effective operation of the brakes and castors.

Safety Precautions

Remember in a shower environment the floors are wet and slippery. Be aware that if the user leans far forward in the chair and takes the weight off the rear wheels the shower chair will tend to slide backwards even with the brakes applied.

Do not sit on the front edge of the seat as this will also tend to tip the chair forward.

Maintain an upright posture and lean against the backrest.


Warranty Information

The Homecraft mobile shower chairs are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc. The manufacturer will not accept responsibility for any damage or injury caused by misuse or nonobservance of the instructions set out above.

Caution

- Always take extra care when getting into or out of the shower chair.
- Brakes should always be applied with the castors facing forward (fig. 1) when entering or leaving the chair.
- With the brakes applied the shower chair may still slide on a smooth floor or if it is tipped forward by the user.
- Supervision is essential while getting on or off the shower chair.

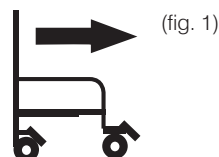(fig. 1)

Maximum user weight

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy.

Please retain these instructions for future reference.

Performance Health International Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
Tel: +44 1623 448 706
www.performancehealth.co.uk

Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France

Distributed in the United States By:

Performance Health Supply, LLC
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
+1800-323-5547
www.performancehealth.com

Distributed in Australia By:

Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Penulwuy NSW, 2145, Australia
1300 473 422
www.performhealth.com.au

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