KING WALL MOUNTED SEATS

WITH/WITHOUT LEGS

EN USER MANUAL
DE BENUTZERHANDBUCH
FR MANUEL DE L'UTILISATEUR
ES MANUAL DE USUARIO
IT MANUALE D'USO
PT MANUAL DO USUÁRIO
SL NAVODILA ZA UPORABO
HR KORISNIČKI PRIRUČNIK
HU HASZNÁLATI UTASÍTÁS
RO MANUAL DE UTILIZARE

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WITH/WITHOUT LEGS

INTENDED USAGE:

The products of the *Profilo KING* line are meant to be used while showering and using the bathroom in general, allowing the user to achieve greater independence and reducing muscle fatigue due to standing position as well as slip and injury risks. The folding mechanism makes these products suitable even for small bathrooms and helps the user keep the shower tray tidy and free from any obstacles. The seats are available in different sizes and with or without legs. The armrests make the user of the seats safer and more comfortable.

INDICATIONS AND CONTRAINDICATIONS:

The products of the *Profilo KING* line are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty maintaining the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The products must not be used if the user has problems related to balance or perception disorders.

SAFETY WARNING AND INDICATIONS:

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the product is correctly assembled.
- During assembly, strictly adhere to what is shown in the User Manual.
- The risk of finger injuries may increase during the mounting process.
- Do not modify the product and its components in any way.
- Do not use the product if damaged or altered in any way.
- After use, the product should be folded in its non-use configuration.
- Do not use the product for a different purpose from the intended one.
- The product must not be stored outdoors.

MAINTENANCE INFO:

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

REUSE REQUIREMENTS:

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

INCIDENT REPORTING:

If there is a serious incident involving this device, contact the manufacturer of this product at info@kingbathrooms.com and the responsible authority in your country.

ITEM CODE	DESCRIPTION	PAGE
KING-PWS-KN	Profilo wall mounted seat King	44
KING-PWSL-KN	Profilo wall mounted seat with legs King	45
KING-PWSS-KN	Profilo wall mounted seat short King	44
KING-PWA-KN	Profilo wall mounted armrests King	47
KING-PWB-KN	Profilo wall mounted backrest King	47

ICON	DESCRIPTION
	Made in Italy
	Easy to assembly
	Max User Weight
	2 Years Warranty

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LM Global Design Ltd.