
User manual

Swift Commode

etac[®]
Creating Possibilities

78323 - 2025-07-29 - Version 20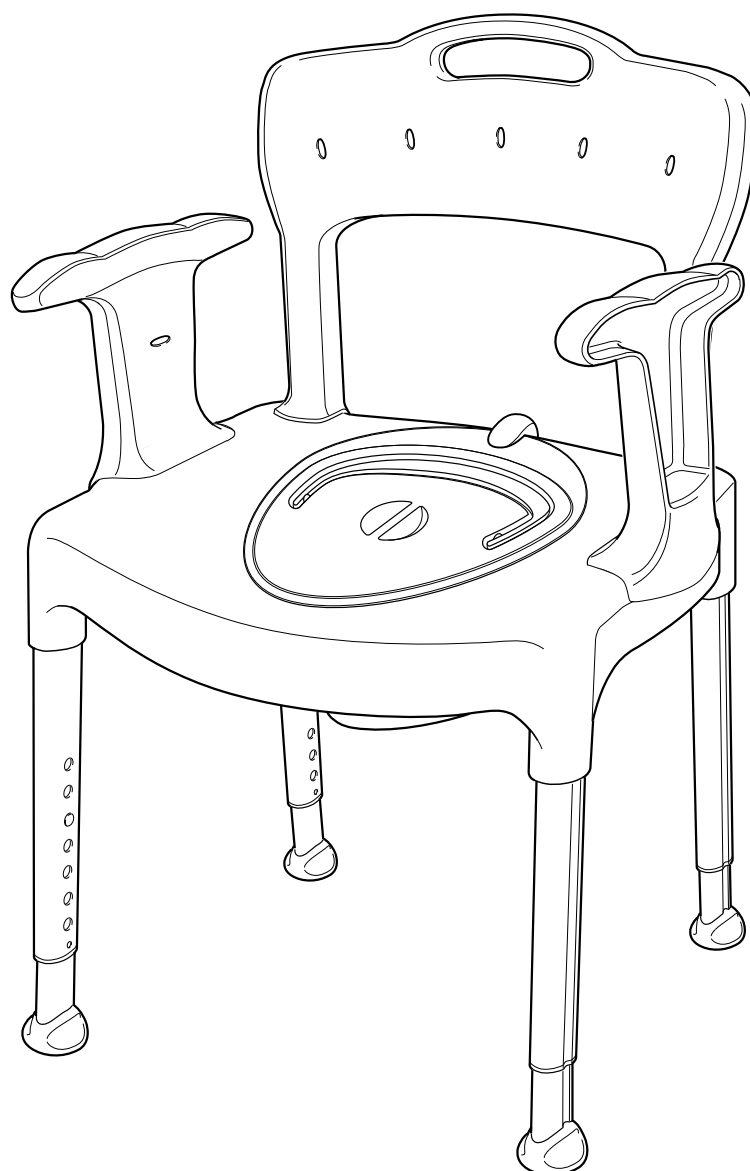

sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar


www.etac.com

MD


sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar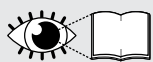


5 5 6 7 8 8 9 10 11 11 12 13 14 15 15 16 17 18 19 19 20 22


A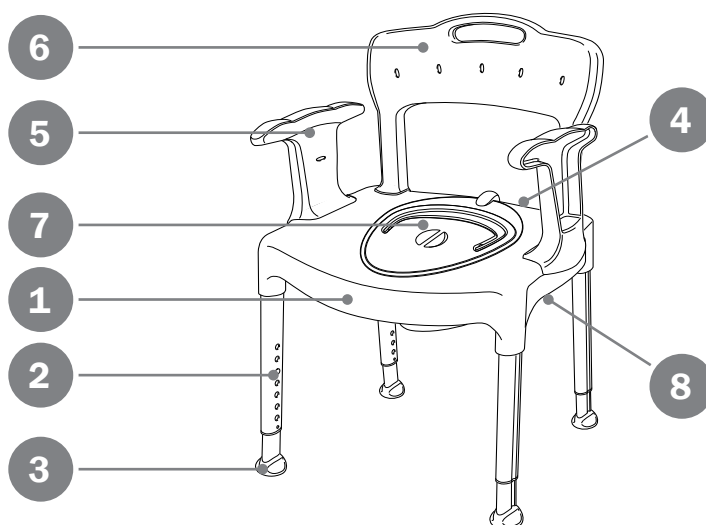

B






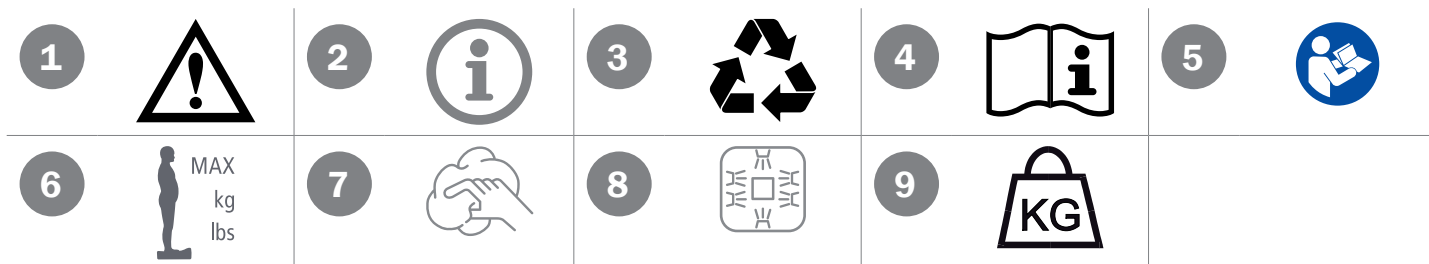
C




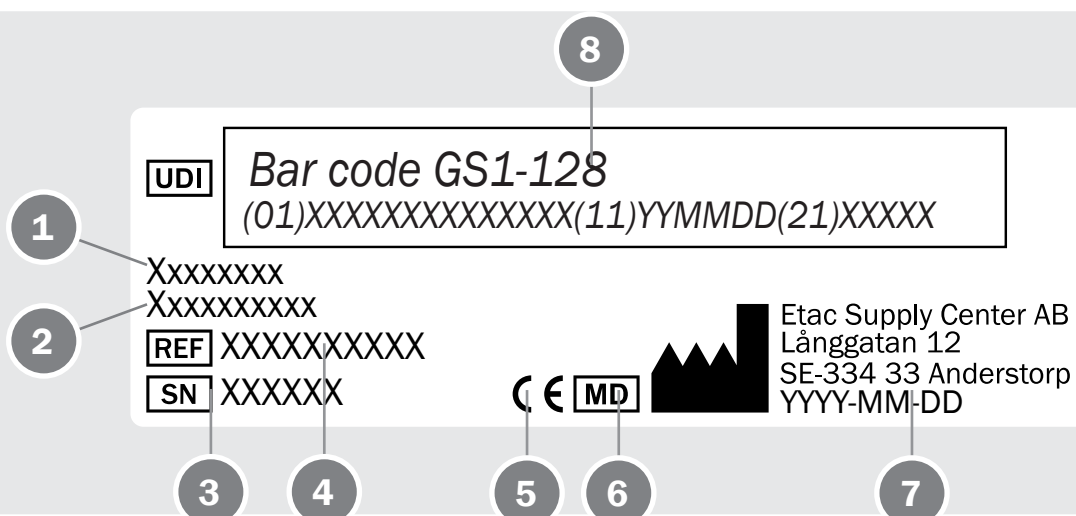
sv en da de it nl no bg cs el es et fi fr hr is lt ms pl pt sk ar


D

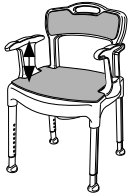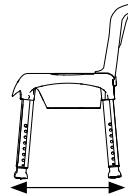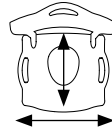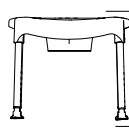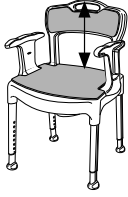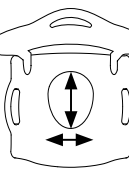								
Etac Swift Commode	210 mm 8 1/4"	450 mm 17 3/4"	540 mm 21 1/4"	800-950 mm 31 1/2"-37 1/2"	530-550 mm 20 3/4"- 21 3/4"	540 x 410 mm 21 1/4" x 16 1/4"	420-570 mm 16 1/2"- 22 1/2"	6,2 kg 14 lbs
Etac Swift Commode basic	210 mm 8 1/4"	450 mm 17 3/4"	540 mm 21 1/4"	800-950 mm 31 1/2"-37 1/2"	530-550 mm 20 3/4"- 21 3/4"	540 x 410 mm 21 1/4" x 16 1/4"	420-570 mm 16 1/2"- 22 1/2"	5,6 kg 12 lbs
								
Etac Swift Commode	380 mm 15"	448 mm 17 3/4"	300 x 215 mm 11 3/4" x 8 1/2"	160 kg				



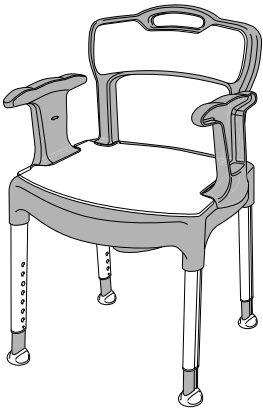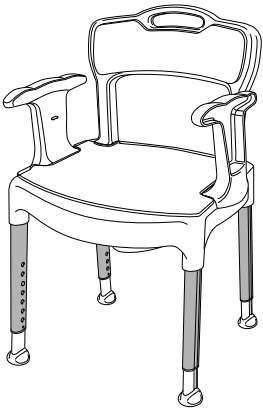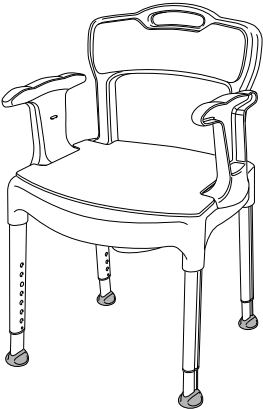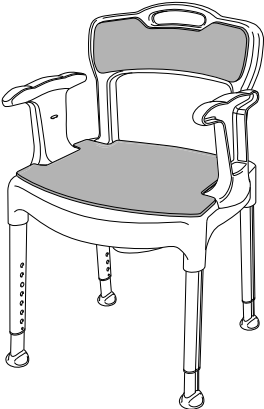
			
PP	Alu	TPE	PE

E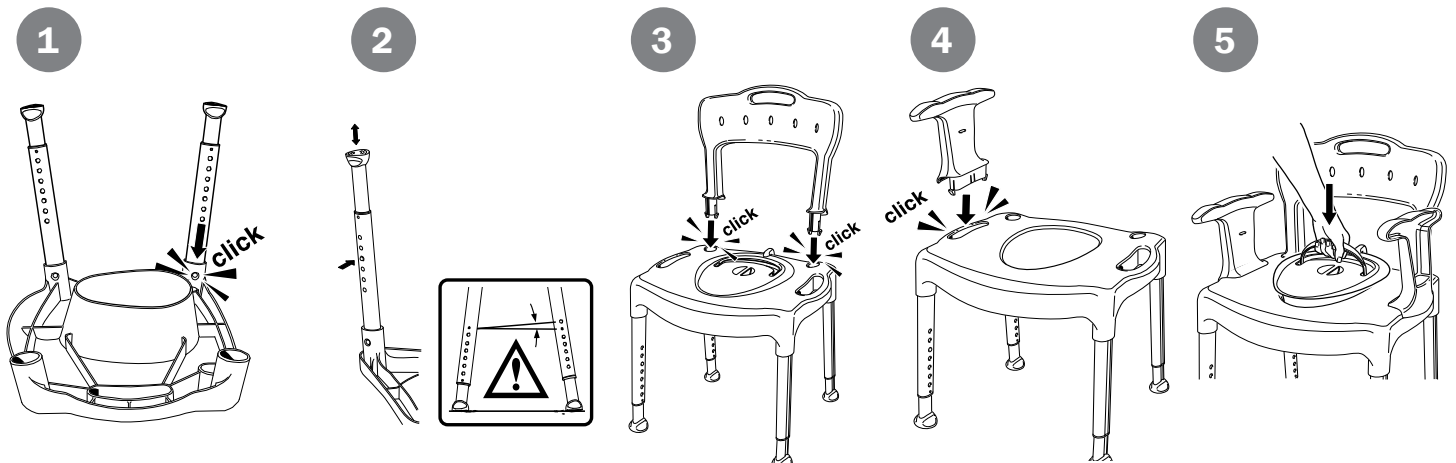


F




1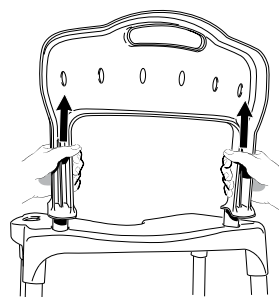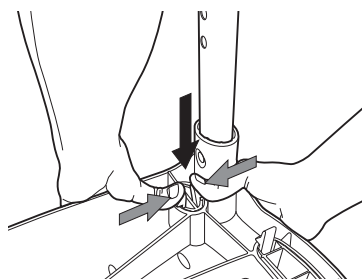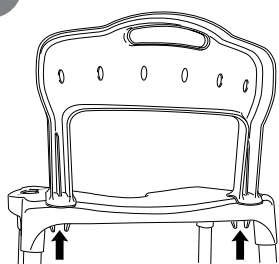




2

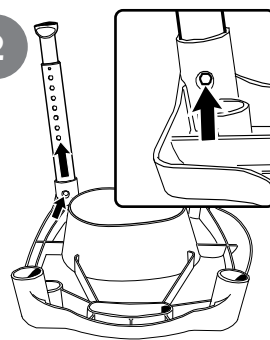3

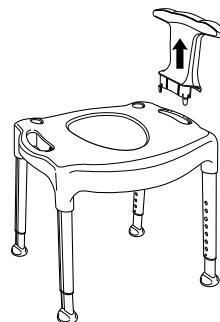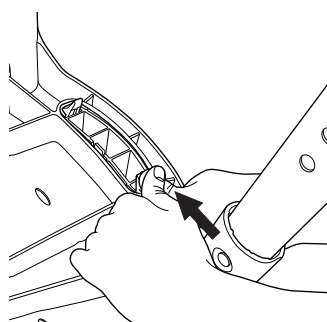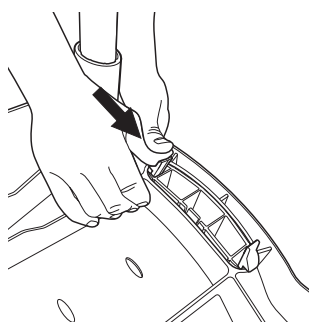4

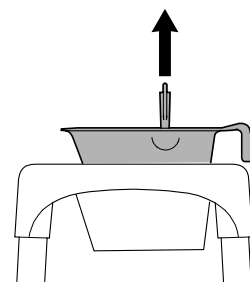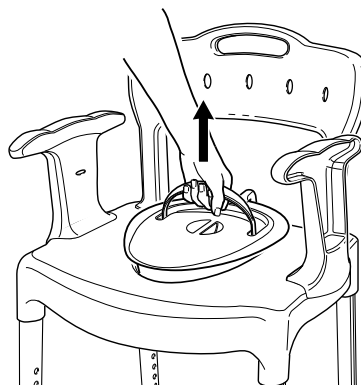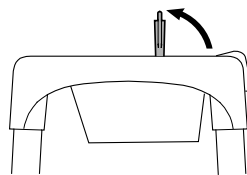
G

1

2

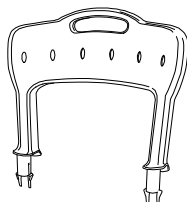
3


4

5

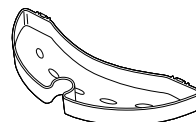6

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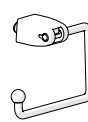Thank you for choosing an Etac product.

In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

In this manual the user is the person with disability for whom the assistive device is intended. The assistant is the person supporting the user.

Device description

Swift Commode is a shower commode chair with detachable armrests and backrest. It is height adjustable to be used as a stand-alone toilet, over the toilet or in the shower. It can be used with one, two or without arm supports. If the arm supports are not used, the opening enables a stable grip and support.

Intended purpose

Swift Commode (hereafter also referred to as "the device" or "the product"), is a medical device intended for alleviation of or compensation for a functional impairment due to an injury or disability. The device is designed to be used when performing hygiene tasks in the shower, at a sink, or over the toilet, as well as for transfers to and from the hygiene room.

Intended user group

The user group for the device is based on the individual's functional ability and not on a specific diagnosis, health condition or age. They are intended for individuals with a height of 146 cm or more, or a mass of 40 kg or more.

Secondary users of the devices are caregivers providing assistance and clinicians/technicians who set up the device.

Intended environment

The device is intended for indoor use at home environments or institutions and are suitable for use in bathrooms, but not swimming pools or similar corrosive environments.

Intended application

The device is intended for short-term and long-term use and can be applied several times per day.

The device is intended for use with intact skin. The device is intended for refurbishment and reuse.

Expected service life

The expected service life is 10 years. For complete information regarding the service life of the device, see www.etac.com.

Indications

Indication for use is disability, including, but not limited to, an individual with physical disabilities who needs backrest and/or armrests to maintain a sitting position and assistance with transfers to/from the chair.

Contra indications:

There are no known contraindications.

Warnings

Warnings describing a risk element for a specific action or setting of the device can be found in the relevant section.

Check user sensitiveness for pressure ulcer before use

A defective device must not be used. Risk of injury.

Do not apply load to only one arm support. Risk of injury if the device tips over.

A heated device can cause burns.

Ignoring the instructions for use may result in injury.

Declaration of conformity

The device meets the requirements of the Medical Device Regulation (EU) 2017/745.

The device meets the requirements of ISO 10993-1, EN ISO 21856.

Pre-purchase information

Pre-purchase information can be found at www.etac.com.

Adverse event

In case of an adverse event occurred in relation to use of the device, it must be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Customization

is everything that goes beyond the manual's instructions and settings. Device specially adapted by the customer must not retain Etac's CE marking. Etac's warranty expires.

Device description [Figure A](#)

1. Seat **2.** Leg **3.** Ferrule **4.** Arm support **5.** Back support **6.** Swift Commode pan including lid **7.** Product label **8.** Item no label*

Symbols, labels [Figure B](#)

Symbols in the manual and on the device:

1. Warning, precaution or limitation. **2.** Useful advice and tips. **3.** Material for recycling. **4.** Refer to user manual. **5.** Refer to user manual (blue and white). **6.** Maximum user weight (see technical data) **7-8.** Cleaning (see maintenance) **9.** Product weight

Explanation item no label: [Figure C](#)

1. Product name **2.** Product description **3.** Serial number **4.** Item number **5.** CE marked **6.** Medical device **7.** Manufacturer and date of manufacture **8.** Barcode acc. to GS1-128 GTIN-14 and serial number*

*The device's date of manufacture can be read from the barcode on the device. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Technical data [Figure D](#)**Installation [Figure E](#)**

Maximum 1 hole height difference between the legs is allowed.

Dismantling [Figure F](#)

1. Back support **2.** Leg **3.** Arm support **4.** Pan

Maintenance

Clean the device with a solvent-free detergent

which has a pH of 5-9, or with a 70% disinfectant solution. Can be decontaminated at max. 85 °C if so required.

Constituent materials are corrosion-resistant. 5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Storage and disposal

The device should be stored indoors in a dry place at a temperature above 5 °C. If the device has been stored for a long time (more than four months), its function must be checked by an expert before use.

To be recycled according to national regulations.

Accessories [Figure G](#)

1. Swift Commode pan including lid **2.** Back support **3.** Soft seat **4.** Soft back **5.** Soap dish/Lid support/Shower head holder **6.** Toilet paper holder

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**Etac Ltd.**
**Promefa AG**
**Etac Supply Center AB**