Intended Use

The Homecraft Mobile commode is designed for use over a toilet and may be used to transport persons over short distances. Designed for those with a Disability or physical impairment and who require assistance with toileting. For indoor use only.

Model Type

Chrome plated steel adjustable height mobile commode chairs, comes with swinging drop down and detachable armrests, detachable back, 4 × 5" brake castors, plastic commode seat with square bowl, upholstered overseat and backrest, built in pushing handle.

REF	Description
091081124	460mm (18") Seat Width
091438936	500mm (20") Seat Width
091438134	560mm (22") Seat Width

Operation

Caution

Maximum user weight

These products should always be used under supervision.

Please ensure all four castors are securely fixed before each usage


Brakes/Castors

The Homecraft Deluxe Chrome Plated range of mobile commodes is fitted with 4 fully braked castors. Each castor can be braked by pushing down the lever on the castor. This prevents the wheels from rotating or swivelling. To release the brake the lever should be pushed upward.

Caution

Brakes should always be applied with the castors facing forward (**fig. 1**) when entering or leaving the chair. With the brakes applied the commode may still slide on a smooth floor or if it is tipped forward by the user. Supervision is essential while getting on or off the commode.

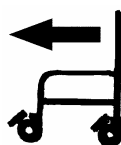(fig. 1)

Bowl Removal

The commode bowl is removed by sliding out from the back of the seat.

Armrests

These models have two armrests, which slot into brackets by the side of the chair. The arms can be removed by pressing the lever towards the arm, raising the front side of the arm and then releasing the arm at the rear by pressing on the spring button. When re-fitting the arm, ensure that the spring button is locked in place. To lower the armrests only,

press the lever towards the arm, raise the arm and then lower to one side until it is resting on the support bar.

Caution


- Ensure the arms are securely slotted into the brackets before use.
- Do not attempt to lift the commode by the arms.

Footrests

The range of mobile commodes are fitted with hook on footrests as standard. To adjust height of footrests, loosen the hexagonal bolt on bottom of tube until the footrest can move smoothly, slide down until the height required and tighten the hexagonal bolt until the footrest is rigid. For access, the footrest plates can be swivelled up vertically.

Fitting of Detachable Backrest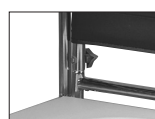

The backrest is fitted by inserting the back into the frame and tightening up the screw with the hand knob, ensuring that the screw head is recessed in the hexagonal shape on the inside of the tube and the hand knob and washer is fitted on the outside.

Perfection Pan (091081876) & pan rack (091184217) optional extras

All commodes can be fitted with Perfection Pan Rack.


To fit the pan rack slide it onto the runners under the seat (091456441) and hook the rear crossbar over the plastic clip at the back of the commode seat. The rack will take the Perfection Pan and a disposable pan.

091456441

091081876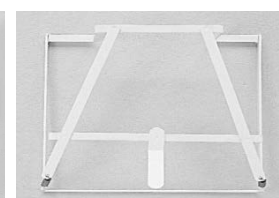

091184217

Height Adjustment

To adjust the height take the load off the castors then press in the two buttons that lock-in the extension legs. Turn the leg slightly then pull the leg to the new leg setting. The two buttons should pop out fully to secure the leg. Make sure all four legs are set at the same height.

Maintenance

- The Homecraft Deluxe chrome Plated Mobile Commodes are designed for minimum maintenance and easy, modular replacement of parts for long life. It is manufactured from chrome plated steel.
- To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.
- Do not use harsh abrasives.
- To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry. Do not use any decontamination process that involves immersion.
- Daily, before use, check the effective operation of the brakes and the secure fixing of the castors to the legs. Check that the commode is stable, i.e. all four castors are touching the floor and none of the legs are bent. Check that the commode arms are located correctly.

- If you find wear or damage to the commode, do not use.
- The chair is designed to allow damaged or broken parts to be changed easily. If any parts are damaged please contact the supplier of the chair or Performance Health direct for replacement parts and fitting instructions.

Warranty Information

The Homecraft Deluxe Chrome Plated Mobile Commodes are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy.

Please retain these instructions for future reference.

Performance Health International Ltd.
Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK
UK:
Tel: 03448 730 035
www.performancehealth.co.uk
International:
Tel: +44 1623 448 706

Distributed in the USA by:
Performance Health Supply, LLC
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012 USA
+1 800-323-5547
www.performancehealth.com

Distributed in Australia by:
Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Pemulwuy NSW, 2145, Australia
1300 473 422
www.performancehealth.au

Distributed in Canada By:
Performance Health Canada, Inc.
2810 Argenta Road, Unit 2
Mississauga, ON L5N 8L2, Canada
+1 800-665-9200
www.performancehealth.ca

CEpartner4U
Amerlandseweg 7
3621 ZC Breukelen
The Netherlands
www.cepartner4u.com

Distribué en France par:
Sissel France SAS
14 rue des Grandes Bosses
44220 Coueron, France
+33 (0)2 40 16 98 76
www.performancehealth.fr

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