Model Types

18" wide chrome plated frame, plastic commode seat with square bowl, upholstered over-seat and backrest, 190 kg (30 stone) maximum user weight.

Operation

Fitting of Detachable Backrest

The backrest is fitted by inserting the back into the frame and tightening up the screw with the hand knob, ensuring that the screw head is recessed in the hexagonal shape on the inside of the tube and the hand knob and washer is fitted on the outside.

091468677 Hand Knob

CAUTION

These products should always be used under supervision.

Bowl Removal - The commode bowl is removed by sliding out from the back of the seat.

Armrests

The armrests slot into brackets by the side of the chair. The arms can be removed by pressing the lever towards the arm, raising the front side of the arm and then releasing the arm at the rear by pressing on the spring button. When re-fitting the arm, ensure that the spring button is locked in place. To lower the armrests only, press the lever towards the arm, raise the arm and then lower to one side until it is resting on the support bar.

091468685
Armrest Release Clip

WARNING!

– Ensure the arms are securely slotted into the brackets before use.
Do not attempt to lift the commode by the arms.

Perfection Pan & Pan Rack

All commodes can be fitted with Perfection Pan Rack (091184217).

To fit the pan rack slide it onto the runners under the seat and hook the rear crossbar over the plastic clip at the back of the commode seat.


The rack will take the Perfection Pan (Days Healthcare part no. 640) and the disposable pan.

Seat Assembly

091081876 Perfection Pan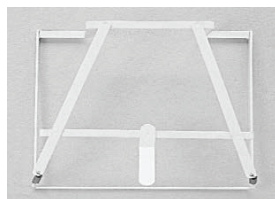

091184217 Perfection Pan Rack

Maintenance

The Homecraft Adj. Hgt. Static commode is designed for minimum maintenance and easy, modular replacement of parts for long life. It is manufactured from chrome plated steel.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water.

Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry. Do not use any decontamination process that involves immersion.

Daily, before use, check that the commode is stable and none of the legs are bent. Check that the commode arms are located correctly.

The chair is designed to allow damaged or broken parts to be changed easily. If any parts are damaged please contact the supplier of the chair for replacement parts and fitting instructions.

Warranty Information

The Homecraft Adj. Hgt. Static commode is warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc. The manufacturer will not accept responsibility for any damage or injury caused by misuse or non observance of the instructions set out above.

Maximum user weight

Serious events experienced from using this product should be reported immediately to Performance Health and to the local Competent Authority.

Symbols Glossary located at <http://www.performancehealth.com/medical-device-glossary>.

These user instructions are available in large print, please contact Performance Health Customer Services for a copy.

Please retain these instructions for future reference.

Performance Health International Ltd.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire, NG17 2HU, UK
Tel: +44 1623 448 706
www.performancehealth.co.uk

Performance Health France
13 rue André Pingat CS 10045
51 724 Reims Cedex France

Distributed in the United States By:

Performance Health Supply, LLC
W68 N158 Evergreen Blvd
Cedarburg, WI 53012, USA
+1800-323-5547
www.performancehealth.com

Distributed in Australia By:

Ausmedic Australia, Trading as
Performance Health ANZ
Unit 3, 3 Basalt Road
Pemulwuy NSW, 2145, Australia
1300 473 422
www.performhealth.com.au

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