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General Information: schülke wound pad blue is an elastic, expanded polyurethane (PUR), which is almost always pervious to air and liquids (e.g. water) - imagine a three-dimensional sieve. It has a coarse texture and a relatively rough surface. schülke wound pad blue may display a yellowish discolouration, however this does not affect the quality.

Intended Use: Promotion of wound healing in sterile application by cleaning, covering or filling wounds or in non-sterile application as a secondary dressing (without direct wound contact) in wound treatment or in concomitant wound treatment, e.g. for pressure relief, or preventively as pressure and friction protection.

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Adverse Effects, Side Effects, Residual Risks: **Wound cleaning:** mechanical debridement may cause pain, even with schülke wound pad blue sterile. **Wound treatment:** because of its cell structure a vascularisation (e.g. of granulation tissue) into the material is possible. You can prevent this with a timely dressing change, after 1-4 days using schülke wound pad blue sterile. On passive wounds or parts of the body that receive poor blood supply, pain may occur after application. This occurs when the blood circulation is stimulated by the mechanical stimulus of the material to such an extent, that previously diminished or reduced pain returns. Initially the pain can be intense, however it is usually regulated after a few hours to days. If this effect is not desired, you should forgo its application. In case of an "open wound care" without additional cover you have to take into consideration that there exists barely protection against the ambient atmosphere. Too much pressure due to the material's prior tension, external influence, or suction may lead to compression of vessels and therefore to pressure ulcers. If hypergranulation occurs, please treat it appropriately and switch to other dressing material groups, if necessary. schülke wound pad blue is a very simple product and its effects are due to its structure. It does not contain nor emit any active compounds. With the correct application of schülke wound pad blue there are no known negative side effects, nor incompatibility or interactions with medicine. Typical side effects may include reddening of the skin upon contact and initial "itching". More intense side effects may include return of sensitivity due to stimulation of local blood circulation. Its effects are only physical and end almost immediately after discontinuation of application.

Please report any incidents that have occurred in connection with our dressing material schülke wound pad blue sterile to us or to the competent authority of the member state in which the incident occurred.

schülke -

Contraindications: Tumour wounds and untreated osteomyelitis. Not suitable for contact with organs, contact to exposed blood vessels or anastomotic areas with the danger of damage, or contact with nerves.

Patient Target Group: Children (> 2 -16 years of age) and adults (> 16 years of age) with acute, chronic, post-surgical wounds/wound healing disturbances and thermal wounds.

User/Practitioner: Application must always be done by healthcare professionals. The decision about the particular kind of application of schülke wound pad blue sterile is determined by healthcare professionals and is adapted and appropriated in every single case.

Capability Characteristics and Clinical Benefit: With the sterile wound dressing schülke wound pad blue you can clean, cover or pack wounds. The product can be used both for mechanical debridement in all phases of wound healing and for wound treatment during the phase of exudation and granulation. The expected clinical benefit present itself in a very effective wound cleaning and wound bed conditioning.

Application of schülke wound pad blue sterile for Wound Treatment: schülke wound pad blue sterile is suitable for single use only. Sterility is ensured only by intact package. The product must not be used after the expiration date and is not suitable for reconditioning or resterilisation due to its potential for contamination. schülke wound pad blue sterile must not continuously remain in the wound or on the body. The application may be repeated over a period up to 30 days, always with a new device. Dressing change after 1-4 days, depending upon the indication. Please consider that schülke wound pad blue sterile must not have direct contact to organs, exposed blood vessels, anastomotic areas or nerves. It may have direct contact to mucosa. Any applications in contact, connection or combination with any additional preparations, drugs, solutions, ointments etc., are not guaranteed by us. Applications in combination with additional mechanical, electrical or electronical apparatus or aids are not guaranteed by us. Because of its open pores, schülke wound pad blue shows no suction effect. If you use it as a wound dressing, you need to combine it with a suction source or an absorbent wound pad (e. g. schülke wound pad white sterile, superabsorbents, foil and Redon bottle, NPWT system).

Wound Cleaning / Mechanical Debridement: schülke wound pad blue sterile is suited for mechanical cleaning of nearly every kind of wound (see contraindications) and during all wound phases. It is highly effective in removing very persistent, viscous, and/or moist necrotic coatings. It may collect plenty of detritus. The wound and the edge of the wound can be treated with it. For this treatment schülke wound pad blue sterile remains in the wound only for a short time to clean it. You can treat the wound with either the twisting or the wiping technique. Please ensure that the wiping direction must always be from inside to outside. Illustrations of the application techniques can be found on page 26 of this instructions for use.

English

Deutsch

Français

Italiano

Polski

Čeština

English

Deutsch

Français

Italiano

Polski

Čeština

EN schülke wound pad blue sterile
Instructions for Use

Important: please note that schülke wound pad used for wound cleaning must not remain in the wound. If the proceeding wound dressing is to be made with schülke wound pad, a new wound pad must be applied.

Wound Treatment: schülke wound pad blue sterile is usually applied directly onto or into the wound without any further steps or products (e.g. ointments or irrigation). Additional measures may negatively affect the desired result. By the described open structure of schülke wound pad blue sterile wound irrigations are possible in spite of the filled wound. Because of its rough surface the decision about the particular kind of application of schülke wound pad blue is determined by healthcare professionals and is adapted and appropriated in every single case.

Dressing composition: if the wound dressing composition consists of several sheets or pieces, please count the number of used dressing pieces and note it in the wound documentation, to ensure the removal of all applied pieces. During application and dressing change friction of the wound, wound bed or skin should be avoided as far as possible to prevent causing any or additional lesions. Dressing change frequency: one time every 1-4 days. Dressing technique: each case should be evaluated and adapted accordingly. Ensure that during dressing change, the wound is left uncovered for the shortest possible amount of time in order to avoid cooling down and to keep risk of re-infection low. Illustrated instructions of the application technique for varying types of wounds are shown on page 27.

Storage, Transport, Shelf Life: Store schülke wound pad blue sterile in a dry area that is protected against UV Radiation. The original packaging offers protection against humidity and UV radiation due to the filtering plastic film, and therefore storage is only permitted in the original packaging. The product may display a yellowish discolouration, especially if improperly stored, however this does not affect the quality. Special transport conditions (e.g. cooling) are not necessary. The product is stable for 36 months after the date of manufacture. The date of manufacture and expiration can be found on the product label of the original packaging or on the sterile packaging (peel-pack) of the individual product.

Disposal / Environmental Relevance: In addition to its accredited quality (QM system for medical devices), LIGAMED® medical Produkte GmbH also documents environmental awareness by validation according to EC regulation and participation in „Umwelt-pakt Bayern“ (Bavarian Environmental Understanding). We will send you our current certificates upon request. All schülke wound pad products are single-substance-materials made of polyurethane, consisting of approx. 97% air and can be disposed of with ordinary household waste. Disposal of the packaging is internationally regulated in different ways. In Germany it is regulated by a licensed company.

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General Information: schülke wound pad red is an elastic, expanded polyurethane (PUR), which is almost always pervious to air and liquids (e.g. water) - imagine a three-dimensional sieve. It has a coarse texture and rough surface. **schülke wound pad red may display a yellowish discolouration, however this does not affect the quality.**

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Wound cleaning: mechanical debridement may cause pain, even with schülke wound pad red sterile.

Wound treatment: because of its cell structure a vascularisation (e.g. of granulation tissue) into the material is possible. You can prevent this with a timely dressing change, after 1-4 days using schülke wound pad red sterile. On passive wounds or parts of the body that receive poor blood supply, pain may occur after application. This occurs when the blood circulation is stimulated by the mechanical stimulus of the material to such an extend, that previously diminished or reduced pain returns. Initially the pain can be intense, however it is usually regulated after a few hours to days. If this effect is not desired, you should forgo its application. In case of an "open wound care" without additional cover you have to take into consideration that there exists barely protection against the ambient atmosphere. Too much pressure due to the material's prior tension, external influence, or suction may lead to compression of vessels and therefore to pressure ulcers. If hypergranulation occurs, please treat it appropriately and switch to other dressing material groups, if necessary. schülke wound pad red is a very simple product and its effects are due to its structure. It does not contain nor emit any active compounds. With the correct application of schülke wound pad red there are no known negative side effects, nor incompatibility or interactions with medicine. Typical side effects may include reddening of the skin upon contact and initial "itching". More intense side effects may include return of sensitivity due to stimulation of local blood circulation. Its effects are only physical and end almost immediately after discontinuation of application.

schülke -

Please report any incidents that have occurred in connection with our dressing material schülke wound pad red sterile to us or to the competent authority of the member state in which the incident occurred.

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User/Practitioner: **Application must always be done by healthcare professionals.** The decision about the particular kind of application of schülke wound pad red sterile is determined by healthcare professionals and is adapted and appropriated in every single case.

Capability Characteristics and Clinical Benefit: With the sterile wound dressing schülke wound pad red you can **clean, cover or pack wounds**. The product can be used both for **mechanical debridement in all phases of wound healing** and for **wound treatment during the phase of exudation and granulation**. The expected clinical benefit present itself in a very effective wound cleaning and wound bed conditioning.

Application of schülke wound pad red sterile for Wound Treatment: schülke wound pad red sterile is suitable for single use only. Sterility is ensured only by intact package. The product must not be used after the expiration date and is not suitable for reconditioning or resterilisation due to its potential for contamination. schülke wound pad red sterile must not continuously remain in the wound or on the body. The application may be repeated over a period up to 30 days, always with a new device. Dressing change after 1-4 days, depending upon the indication. Please consider that schülke wound pad red sterile must not have direct contact to organs, exposed blood vessels, anastomotic areas or nerves. It may have direct contact to mucosa. Any applications in contact, connection or combination with any additional preparations, drugs, solutions, ointments etc., are not guaranteed by us. Applications in combination with additional mechanical, electrical or electronical apparatus or aids are not guaranteed by us. Because of its open pores, schülke wound pad red shows no suction effect. If you use it as a wound dressing, you need to combine it with a suction source or an absorbent wound pad (e. g. schülke wound pad white sterile, superabsorbents, foil and Redon bottle, NPWT system).

Wound Cleaning / Mechanical Debridement: schülke wound pad red sterile is suited for mechanical cleaning of nearly every kind of wound (see contraindications) and during all wound phases. It is highly effective in removing very persistent, viscous, and/or moist necrotic coatings. It may collect plenty of detritus. The wound

English

Deutsch

Français

Italiano

Polski

Čeština

English

Deutsch

Français

Italiano

Polski

Čeština

EN schülke wound pad red sterile
Instructions for Use

and the edge of the wound can be treated with schülke wound pad red sterile. For this treatment schülke wound pad red sterile remains in the wound only for a short time to clean it. You can treat the wound with either the twisting or the wiping technique. Please ensure that the wiping direction must always be from inside to outside. Illustrations of the application techniques can be found on page 26 of this instructions for use. Important: please note that schülke wound pad used for wound cleaning must not remain in the wound. If the proceeding wound dressing is to be made with schülke wound pad, a new wound pad must be applied.

Wound Treatment: schülke wound pad red sterile is usually applied directly onto or into the wound without any further steps or products (e.g. ointments or irrigation). Additional measures may negatively affect the desired result. By the described open structure of schülke wound pad red, wound irrigations are possible in spite of the filled wound. Because of its rough surface the decision about the particular kind of application of schülke wound pad red is determined by healthcare professionals and is adapted and appropriated in every single case.

Dressing composition: if the wound dressing composition consists of several sheets or pieces, please count the number of used dressing pieces and note it in the wound documentation, to ensure the removal of all applied pieces. During application and dressing change friction of the wound, wound bed or skin should be avoided as far as possible to prevent causing any or additional lesions.

Dressing change frequency: one time every 1-4 days.

Dressing technique: each case should be evaluated and adapted accordingly. Ensure that during dressing change, the wound is left uncovered for the shortest possible amount of time in order to avoid cooling down and to keep risk of re-infection low. Illustrated instructions of the application technique for varying types of wounds are shown on page 27.

Storage, Transport, Shelf Life: Store schülke wound pad red sterile in a dry area that is protected against UV Radiation. The original packaging offers protection against humidity and UV radiation due to the filtering plastic film, and therefore storage is only permitted in the original packaging. The product may display a yellowish discolouration, especially if improperly stored, however this does not affect the quality. Special transport conditions (e.g. cooling) are not necessary. The product is stable for 36 months after the date of manufacture. The date of manufacture and expiration can be found on the product label of the original packaging or on the sterile packaging (peel-pack) of the individual product.

Disposal / Environmental Relevance: In addition to its accredited quality (QM system for medical devices), LIGAMED® medical Produkte GmbH also documents environmental awareness by validation according to EC regulation and participation in „Umweltpakt Bayern“ (Bavarian Environmental Understanding). We will send

schülke -†

you our current certificates upon request. All schülke wound pad products are single-substance-materials made of polyurethane, consisting of approx. 97% air and can be disposed of with ordinary household waste. Disposal of the packaging is internationally regulated in different ways. In Germany it is regulated by a licensed company.

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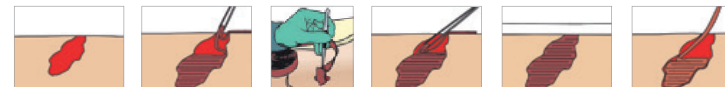
Fertigwarennummer: 146010
 alt: 20004163-C / 305070801-01
 Design Change

Exclusion of liability for the implementation of statutory provisions and color liability. This PDF is not color-coded, it only serves to check text, status and color distribution. Compliance with applicable statutory provisions with regard to font sizes, industry-sector-specific labeling requirements, trademark, competition and licensing laws must be checked and ensured by the customer.

Please use the overprint option for PDF preview.

WHITE | WEISS | BLANC | BIANCO | BIAŁY | BÍLÝ

- EN** LEAFLET
- DE** BEIPACKZETTEL
- FR** BROCHURE
- IT** FOGLIO ILLUSTRATIVO
- PL** ULOTKA
- CZ** LETÁK

schülke ->

English

Deutsch

Français

Italiano

Polski

Čeština

EN schülke wound pad white sterile
Instructions for Use

MD **CE** 0297 (appendix II, class IIb, acc. to regulation 4)

Please read this instruction carefully before use!

General Information: schülke wound pad white is a therapeutically effective PUR foam material with a wide range of applications for wound care and prevention. The therapeutic uses are based on three underlying physical principles:

Mechanical stimulus: Stimulates local blood circulation within the wound and on the skin, thereby improving the supply of nutrients and oxygen in the wound area (activation of the wound) and acts as a preventative on intact skin. Stimulates normal bodily functions on a local level, especially in the case of inactive patients or, where these functions are reduced. The mechanical stimulus lasts up to three days, by which time schülke wound pad white should then have been changed.

Low-pressure tension: Low-pressure adaption to contours and minimised pressure on wound and body.

Specific suction effect: Absorbs excess fluid without a drying-out effect. Creates a moist and warm wound environment, cleans the wound and thus reduces the bacterial load, without sticking to the wound. Averts skin maceration and is skin-friendly.

schülke wound pad white may display a yellowish discolouration, however this does not affect the quality.

Intended Use: Promotion of wound healing in **sterile application by cleaning, covering or filling wounds** or in non-sterile application as a secondary dressing (without direct wound contact) in wound treatment or in concomitant wound treatment, e.g. for pressure relief, or preventively as pressure and friction protection.

Indications: Acute and chronic wounds (e.g. decubital ulcers, ulcer cruris, diabetic foot ulcers). Post-surgical wounds and post-surgical wound healing impairments. Thermal wounds, such as burns, scalds, frostbite.

Adverse Effects, Side Effects, Residual Risks:

Wound cleaning: mechanical debridement may cause pain, even with schülke wound pad white sterile.

Wound treatment: because of its cell structure a vascularisation (e.g. of granulation tissue) into the material is possible. You can prevent this with a timely dressing change, after 1-3 days using schülke wound pad white sterile. On passive wounds or parts of the body that receive poor blood supply, pain may occur after application. This occurs when the blood circulation is stimulated by the mechanical stimulus of the material to such an extent, that previously diminished or reduced pain returns. Initially the pain can be intense, however it is usually regulated after a few hours to days. If this effect is not desired, you should forgo its application. Too much pressure due to the material's prior tension, external influence, or suction may lead to compression of vessels and therefore to pressure ulcers. It may occur that schülke wound pad white sterile sticks to the wound, mostly due to too much air reaching the wound. This means that the schülke wound pad white sterile dressing is not thick enough (it should be 2 cm from skin level).

schülke -

schülke wound pad white sterile can be moistened on the side facing the wound (e.g. with Ringer's solution or wound irrigation solution) to prevent it from sticking. If hypergranulation occurs, please treat it appropriately and switch to other dressing material groups, if necessary. schülke wound pad white is a very simple product and its effects are due to its structure. It does not contain nor emit any active compounds. With the correct application of schülke wound pad white sterile there are no known negative side effects, nor incompatibility or interactions with medicine. Typical side effects may include reddening of the skin upon contact and initial "itching". More intense side effects may include return of sensitivity due to stimulation of local blood circulation. Its effects are only physical and end almost immediately after discontinuation of application. As with all wound dressings, minor skin reactions such as maceration, erythema, secondary infected dermatitis, erysipelas as well as hypersensitivity reactions and pain at dressing change may occur.

Please report any incidents that have occurred in connection with our dressing material schülke wound pad white sterile to us or to the competent authority of the member state in which the incident occurred.

Contraindications: Tumour wounds and untreated osteomyelitis. Not suitable for contact with organs, contact to exposed blood vessels with the danger of damage, or contact with nerves.

Patient Target Group: Children (> 2 - 16 years of age) and adults (> 16 years of age) with acute, chronic, post-surgical wounds/wound healing disturbances and thermal wounds.

User/Practitioner: Application must always be done by healthcare professionals. The decision about the particular kind of application of schülke wound pad white sterile is determined by healthcare professionals and is adapted and appropriated in every single case.

Capability Characteristics and Clinical Benefit: With the sterile wound dressing schülke wound pad white you can **clean, cover or pack wounds**. The product can be used both for **mechanical debridement** and for **wound treatment in all phases of wound healing**. The expected clinical benefit presents itself in a very effective wound cleaning, wound bed conditioning and in the healing of the wounds.

Application of schülke wound pad white sterile for Wound Treatment: schülke wound pad white sterile is suitable for single use only. Sterility is ensured only by intact packaging. The product must not be used after the expiration date and is not suitable for reconditioning or resterilisation due to its potential for contamination. schülke wound pad white sterile must not continuously remain in the wound or on the body. The application may be repeated over a period up to 12 months, always with a new dressing. Dressing change after 1-3 days, depending upon the indication. Please consider that schülke wound pad white sterile must not have direct contact to organs. It may have direct contact to mucosa. Any applications in contact, connection or combination with any additional preparations, drugs, solutions, ointments etc., are not guaranteed by us. Applications in combination with additional mechanical, electrical or electronical apparatuses or aids are not guaranteed by us.

English

Deutsch

Français

Italiano

Polski

Čeština

English

Deutsch

Français

Italiano

Polski

Čeština

EN schülke wound pad white sterile
Instructions for Use

Wound Cleaning / Mechanical Debridement: schülke wound pad white sterile is suited for mechanical cleaning of nearly every kind of wound (see contraindications) and during all wound phases. It is highly effective in removing soft coatings and biofilm. The wound and the edge of the wound can be treated with schülke wound pad white sterile. For this treatment schülke wound pad white sterile remains in the wound only for a short time to clean it. You can treat the wound with either the twisting or the wiping technique. Please ensure that the wiping direction must always be from inside to outside. Illustrations of the application techniques can be found on page 26. Important: please note that schülke wound pad used for wound cleaning must not remain in the wound. If the proceeding wound dressing is to be made with schülke wound pad, a new wound pad must be applied.

Wound Treatment: schülke wound pad white sterile is usually applied directly onto or into the wound without any further steps or products (e.g. ointments or irrigation). Additional measures may negatively affect the desired result. Apply a 1-2cm thick schülke wound pad white sterile to skin-surface wounds, the edges overlapping the wound by 1-2cm. Deeper wounds, pocket-shaped wounds and hollow wounds should be **completely packed with sterile schülke wound pad white**. There must be reliable contact completely covering the wound. To guarantee this, schülke wound pad white sterile can be compressed.

Padding thickness: measuring from the wound base, schülke wound pad white sterile should be at least 2cm thick. The thickness can also be formed using several layers, including the outer covering of the wound. If the wound dressing composition consists of several sheets or pieces, please count the number of used dressing pieces and note it in the wound documentation, to ensure the removal of all applied pieces.

Dressing change frequency: one time every 1-3 days, based on necessity, and as soon as the first sign of secretion becomes visible on the outside of schülke wound pad white sterile. For wounds that heavily suppurate during the cleaning phase, the dressing should also be changed several times daily.

Padding technique: each case should be evaluated and adapted accordingly. Ensure that during dressing change, the wound is left uncovered for the shortest possible amount of time in order to avoid cooling down and to keep risk of re-infection low.

Typical wound cleaning sequence: with chronic wounds the wound-cleaning phase requires approx. 10-20 dressing changes. The intended result is the smallest possible "clean" wound, with low susceptibility to re-infection. Illustrated instructions of the application technique for varying types of wounds are shown on page 27.

Storage, Transport, Shelf Life: Store schülke wound pad white sterile in a dry area that is protected against UV Radiation. The original packaging offers protection against humidity and UV radiation due to the filtering plastic film, and therefore storage is only permitted in the original packaging. The product may display a yellowish discolouration, especially if improperly stored, however this does not affect the quality. Special transport conditions (e.g. cooling) are not necessary. The product is stable for 36 months after the date of manufacture. The date of manufacture and expiration can be found on

schülke ->

the product label of the original packaging or on the sterile packaging (peel-pack) of the individual product.

Disposal / Environmental Relevance: In addition to its accredited quality (QM system for medical devices), LIGAMED® medical Produkte GmbH also documents environmental awareness by validation according to EC regulation and its participation in „Umweltpakt Bayern“ (Bavarian Environmental Understanding). We will send you our current certificates upon request. All schülke wound pad products are single-substance-materials made of polyurethane, consisting of approx. 97% air, and can be disposed of with ordinary household waste. Disposal of the packaging is internationally regulated in different ways. In Germany it is regulated by a licensed company.

Symbols used for labeling:

	Item number		Manufacturer
	Lot number		Production date
	Use by: year / month		Distributor
	Medical device		Sterilised by steam
	Unique device identifier		Disposable product
	Consult instructions for use		Protect from sun and UV radiation
	Do not use if the packaging is damaged		Protect from moisture

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Schulke & Mayr GmbH • Robert-Koch-Str. 2 • 22851 Norderstedt
Schulke & Mayr AG • 8500 Frauenfeld • Tel. +41 44 4665544
Responsible Person UK: Schulke & Mayr UK Ltd. • GB-Sheffield S9 1AT •
Schulke & Mayr UK Ltd. • GB-Sheffield S9 1AT • Tel. +44 114 2543500