Cojines Antiescaras

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(ENG) PRESSURE RELIEF CUSHIONS

Description and intended use:

The cushions are available in various formats (flotation and viscoelastic), sizes and materials as per the following table:

Model	Size	Dimensions (cm)	Description	Max. weight
CPVA220	1	40X40X8 cm	Anatomical viscoelastic cushion with a density of 60kg/m ³	100kg
	2	42X42X8 cm		
	3	42X45X8 cm		
CPVA280	1	42X42X8 cm	Anatomical viscoelastic cushion with a density of 80kg/m ³	130kg
	2	42X45X8 cm		
CPFL214	1	41X41X4,5 cm	Square-shaped flotation cushion	150kg
	2	41X41X6 cm		
CPFL218	1	42X42X4,5 cm	Horseshoe-shaped flotation cushion	150kg
	2	42X42X6 cm		

Note: The covers meet the requirements of the BS7175 standard. They are class CRIB5 fire-retardant and manufactured with a specially treated material to reduce the growth of fungi and bacteria.

Indications:

Prevention of eschars (pressure ulcers) in the sacral gluteal area for people with reduced mobility who spend a lot of time sitting. Pressure relief cushions increase the contact area between the body and the surface of the chair, distributing and reducing pressure in prominent areas.

Warnings:

- Read these instructions carefully before using this cushion. If you have any questions, consult your doctor or the shop that sold you the product.
- The general instructions for use and the specific indications provided by the doctor must always be followed.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- To prevent the onset of pressure ulcers, it is important to use a pressure relief cushion. In addition to this, whenever possible, make regular movements in order to avoid prolonged blood vessel compression and enable good tissue irrigation.
- For reasons of hygiene, the cushion should only be used by one patient. Clean the product regularly and maintain good personal hygiene.
- In the event of incontinence, wash the cover immediately and clean the cushion with a damp cloth. We recommend that you wash the cover regularly.
- Prevent moisture from accumulating on the cover.
- It is advisable to purchase a spare cover to use when the other one is being washed.
- We do not recommend using the pressure relief cushion on a different part of the body to that indicated (sacral gluteal area).
- Make sure that the top of the cushion is not creased.
- Do not place any additional elements between the patient and the cushion (except for clothing).
- Avoid using the cushion on surfaces that are excessively sloped, as this may cause the cushion to move and it will not be stable enough to perform its function correctly.
- Do not let children play with this product.
- Take special care to prevent tears or any other physical damage that may impair the cushion and render

it unusable.

- If you have a liquid flotation model and it leaks, do not ingest the liquid or allow it to come into contact with mucous membranes or the eyes, as it may be harmful.
- Some of the materials used in the product are not flame retardant. Therefore, its use is not recommended in environments with excessive heat, fire or radiation.
- Store horizontally inside the original packaging, in a dry place away from direct light. Do not leave the cushion inside a car in sunlight.
- Although the cushion is not a single-use product, it should only be used by a single patient and only for the purposes indicated in these instructions or by your doctor.
- It is important to remember that although support surfaces are a valuable ally in relieving pressure, under no circumstances are they a substitute for changes to posture.

Note: Any serious incidents involving the product must be reported to the manufacturer and to the competent authority in the Member State where the user and/or patient is located.

Instructions for use:

General considerations to bear in mind to ensure correct use of the cushion:

- Choose a cushion size that is compatible with the chair on which it will be used and the weight of the patient. Qualified healthcare professionals will be able to guide you when selecting a product to suit your needs.
- The non-slip area (i.e. the grey-coloured side of the cover) must be in contact with the surface of the chair.
- On all models, the cover's zip and handle are located on the back of the cushion.
- The black PU side of the cover is the area that supports the body.

Positioning instructions for the cushion:

1. Place the cushion on the seat, making sure the non-slip area is in contact with the surface of the chair.
2. Make sure the rear side of the cushion - the side in contact with the seat - is flat.
3. On all models, the cushion must be positioned so that the cover's handle is at the rear of the seat area.

Product maintenance – washing instructions:

On all models, only the cover can be washed, as per the following instructions:

Cleaning: The outer cover (or the cushion, if necessary) can be cleaned with a damp cloth. Use a dry cloth afterwards to absorb any moisture.

Washing the outer cover: Before washing the cover, remove the stuffing, then close the zip and machine wash at a maximum of 95°C, using a mild soap. The cover can be dried in a dryer, provided the temperature does not exceed 80°C. Do not use bleach or abrasive soaps. Only use a mild soap. Do not iron. Do not dry clean.

SeWhen drying the cover, avoid forming creases. Only place it back on the cushion when it is completely dry. If the cushion's stuffing needs to be cleaned, use a damp cloth and dry immediately. Avoid exposing the cushion to very hot heat sources, such as stoves or radiators, and keep it away from direct sunlight.

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PRIM, S.A.