CARE

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Instructions for use

All product-related serious incidents must be reported to the manufacturer of ANITA Dr. Helbig GmbH, as well as the responsible authority.

Article **REF**

Textile lightweight breast forms with individual volume adjustment (fleece)
1010X6, 1014X, 1018X, 1019X

Textile lightweight breast forms made of foam
1011X

Indication

As initial/lightweight care after breast surgery (ablation/mastectomy, partial surgery)

Contraindications and risks

No contraindications or risk for the use of the products are known or can be expected. In rare cases, allergic reactions may occur. Please contact your doctor or healthcare professional if you are not sure whether you may be allergic, or if you have questions or issues regarding the use of these products.

Purpose/How it works

- Target group: Women after various kinds of breast surgery
- Initial/lightweight care after surgery or during radiation therapy
- Skin-friendly, breathable material, reduces skin irritation on sensitive skin, e.g. during radiation therapy or stresses on the scar area

Materials

- 1010X6 – 36% PU, 35% PA, 24% PES, 5% EL, Fleece: 100% PES
- 1011X – Cover: 100% PES, Foam: 100% PU
- 1014X – Cover: 75% CO, 25% PES, Fleece: 100% PES
- 1018X – Cover: 55% PU, 35% PA, 10% EL, Fleece: 100% PES
- 1019X – Cover: 60% PES, 35% PU, 5% EL, Fleece: 100% PES
- Latex-free
- ANITA products bear the OEKO-TEX® STANDARD 100, 98.0.1439 Hohenstein HTTI mark. It guarantees the absence of harmful or allergenic substances.

Care instructions

1010X6, 1014X, 1018X, 1019X

- Please refer to the sewn-in label for care instructions
- Remove the medical fleece before washing.
- Once the garment has dried, the primary care prostheses can be padded with the medical fleece once again.

1011X

- The foam prosthesis can be cleaned in a 40 °C delicates cycle or washed by hand.
- Lay flat to dry and do not tumble-dry or iron.

User instructions

The right size for the textile primary care prosthesis depends on the cup size of the patient. Primary care prostheses can be worn in the pocket of a special textile. Allow a trained professional to select the correct size for you.

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Technical data

Weight in grams (without packaging or medical fleece)

Sizes	1010X6	1011X	1014X	1018X	1019X
60/65	14,5	12,1	22,0	14,8	14,5
70/75	19,4	16,3	35,5	20,3	19,4
80/85	25,0	23,4	45,2	26,4	25,0
90/95	30,0	30,8	51,5	31,0	30,0
100/105	37,5	36,1	62,5	38,9	37,5
110/115	45,4	49,8	91,5	47,0	45,4

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CARE

Designed and made by ANITA.

Thank you for choosing a high-quality product from ANITA.
The 100% family-owned business has been taking responsibility
for its products, employees and the environment since 1886.

ANITA has a certified quality management system that complies
with DIN EN ISO 9001 and DIN EN ISO 13485 to ensure the quality
of products throughout the entire value chain.

ANITA Dr. Helbig GmbH, Grafenstr. 23, D-83098 Brannenburg, anita.com, anita.d@anita.net

All Anita care breast forms bear the CE mark.