
Instructions for use ESENTA™ Strips / Gebrauchsanweisung – ESENTA™ Streifen / Notice d'utilisation des renforts ESENTA<sup>MC</sup> / Istruzioni per l'uso delle strisce ESENTA™ / Brugvejledning til ESENTA™ kantsikring / Instrucciones de uso de las tiras ESENTA™ / Käyttöohjeet ESENTA™ -kiinnitysteipille / Upute za uporabu traka ESENTA™ / Lietošanas pamācība ESENTA™ strēmeles / Juostelių „ESENTA™“ naudojimo instrukcija / Instruções de utilização das tiras ESENTA™ / Plaastriribade ESENTA™ kasutusjuhend / Инструкции за употреба за ленти ESENTA™ / Bruksanvisning ESENTA™-strips / Gebruiksaanwijzing ESENTA™-strips / Instrukcja stosowania pasków ESENTA™ / Bruksanvisning for ESENTA™ plateforlengere / Návod k použití pásků ESENTA™ / Návod na použitie adhezívnej pásky ESENTA™ / Οδηγίες χρήσης για τις ταινίες ESENTA™ / Noutunariabõbeinagar fyrir ESENTA™ strimla / ESENTA™テープの使用方法 / Használati utasítás – ESENTA™ rögzítőcsíkok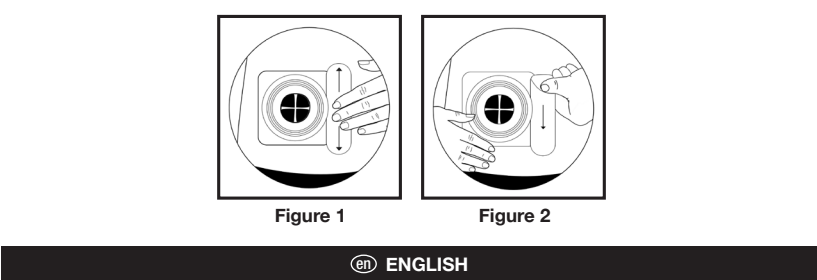

## ENGLISH

#### General information

ESENTA™ Strips are non-sterile hydrocolloid strips, covered with a top layer of a breathable polyurethane. ESENTA™ Strips contain pectin, gelatin, sodium CMC and polyisobutylene. ESENTA™ Strips are not manufactured using natural rubber or latex.

#### Intended use

ESENTA™ Strips are intended to be used as additional fixations for the edges of skin barriers around the stoma.

#### Clinical Benefits:

- Aids adhesion of the baseplate.
- Aids in obtaining a good seal.
- Helps to increase stoma system wear time.

#### Indications

- ESENTA™ Strips are suitable for use on intact skin only.
- To be used with a colostomy, ileostomy or urostomy.

#### Contraindications

Do not use to cover broken skin or open wounds or on individuals who are sensitive to or who have developed sensitivity or allergic reaction to the product.

**Patient target groups:** For individuals with an ileostomy, colostomy or urostomy.

#### Intended Users

Ostomates, healthcare professionals and carers.

#### Warnings and precautions

Should irritation occur, discontinue use, and consult your healthcare professional. This device is for single-use only and should not be reused. Re-use may lead to increased risk of infection and cross-contamination. Physical properties of the re-used device may no longer be optimal for intended use.

#### INSTRUCTIONS/Performance characteristics

#### Preparation

- Choose correct size and form of strips suitable for stoma application.
- The strips should overlap the skin barrier for at least 1.5 to 2cm.
- Clean and dry the skin and ensure it is free from solvent or greasy substances

#### Application

- Remove the release papers.
- Apply the strips to the skin overlapping the skin barrier and smooth down for 30 seconds.

(Figure 1)

- Use one or more strips around baseplate as required.
- Wear as long as comfortable and secure; individual wear time may vary.

#### Removal

Support the skin and gently peel off the strips from the body. (Figure 2)

#### Disposal

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

© 2023 Convatec Inc.

®/™ All trade marks are the property of their respective owners.

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# Effective

| ARTWORK SPECIFICATION                                                                                                          |                                         |
|--------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------|
| Artcode                                                                                                                        | 1738830                                 |
| Artwork Description                                                                                                            | IFU ESENTA MULTILINGUAL                 |
| ICC*                                                                                                                           | 423827, 423824, 423825, 423826          |
| SAP*                                                                                                                           | 1738603,1738600,1738601,1738602         |
| KOM/REF*                                                                                                                       | N/A                                     |
| IS Code*                                                                                                                       | N/A                                     |
| Component Type                                                                                                                 | Insert                                  |
| LR#                                                                                                                            | LR-004860                               |
| Existing Art Code being Revised                                                                                                | N/A                                     |
| REQUIREMENT                                                                                                                    | SPECIFICATION                           |
| Bucket                                                                                                                         | EUR, NAI                                |
| Local Code(s)                                                                                                                  | N/A                                     |
| Size Code / Dieline                                                                                                            | N/A                                     |
| Dimensions                                                                                                                     | Flat 500 x 500 mm; Folded: 125 x 250 mm |
| Color(s) / Colour(s)                                                                                                           | Black                                   |
| Specification Reference(s)                                                                                                     | MT56-024                                |
| Bar Code: UPC (A) or GTIN**                                                                                                    | N/A                                     |
| Bar Code: PZN – Code 39**                                                                                                      | N/A                                     |
| Bar Code: JAN-13**                                                                                                             | N/A                                     |
| * NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade “C” or better. |                                         |
| HISTORY                                                                                                                        |                                         |
| 06-Apr-2023 / 1738830: New artwork created.                                                                                    |                                         |

WI-0010, Ver. 7.0

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© 2023 ConvaTec Inc.