[illegible]

The image displays the PRIM logo, which consists of a stylized 'P' inside a rounded square, with a small star above the vertical stroke. Below the logo is the word 'PRIM' in a bold, sans-serif font. To the left of the contact information is a small icon of a factory with three smokestacks. To the right is the CE mark and a box containing the letters 'MD'. At the bottom, the text 'IU.643.0 25/07/2023 837691' is printed.

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ENG

Description:
Ankle orthosis with two mediolateral stabiliser valves at an anti-rotational angle for the straps for enhanced fit and stabilisation. Right and left versions available.

Indications - Intended use:
For the treatment of chronic ligament deficiency (recurrent sprains). As an injury prevention measure during sporting activities. Post-operative rehabilitation: post-injury or post-surgical immobilisation, first- and second-degree sprains.

Contraindications:

- Do not use on open wounds with swelling.
- Do not use if the area where the orthosis is placed is infected.
- Do not use in the case of skin allergies due to contact with biocompatible materials.
- Do not use if the area starts to redden and/or becomes excessively hot.

Warnings:

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop that sold you the product.
- For the orthosis to serve its purpose, it must be correctly adjusted to the patient's morphology. The first fitting and adjustment must be carried out by qualified personnel (doctor, orthopaedic technician, etc.).
- Always follow these general instructions of use and any particular guidelines indicated by the general practitioner.
- Only a doctor is qualified to prescribe and determine the duration of treatment and carry out subsequent monitoring.
- If you notice any side effects, skin conditions or sensitivity, consult your doctor immediately.
- Your doctor might consider the use of topical creams in conjunction with the orthosis.
- In areas where pressure is applied, the skin should not be damaged or hypersensitive.
- Although the orthosis is not for single use, it should only be used by one patient. The product should not be reused by other patients.
- For the orthosis to function properly, its components must be regularly checked to ensure that they are in good condition. The medical staff responsible for monitoring the treatment may recommend the product to the patient or suggest that it be replaced if it is damaged or worn.
- To ensure that you function correctly, clean the hook-and-loop fastenings regularly, removing any material that may have stuck to them.
- Clean the product regularly and maintain proper hygiene.
- Ask your doctor if you can remove the orthosis when bathing or showering. If you cannot, make sure you cover it or keep it away from the water.
- Do not let children play with this device.

Instructions for proper fitting:

- It is essential that you select the correct size for the patient.
- Open all of the velcro fasteners and release the straps of the ankle support.
- Position the heel over the circular opening of the ankle support (Fig. 1).
- Tighten and fasten the lower section of velcro above the instep (Fig. 2).
- Tighten the upper fasteners and attach the sections of velcro, tightening as required (Fig. 3).
- Fasten the rear strap at the back (Fig. 4).
- Fasten the anti-varus strap, tighten it around the ankle (ensuring compression at all times), then wrap the strap in a helix pattern, starting at the ankle and moving up to the calf, and attach the velcro.
- Check that the ankle support is correctly fitted to the foot (Fig. 6).

Product maintenance - washing instructions:
This product is latex-free. Before washing, fasten all of the velcro fasteners. Wash with a neutral detergent at a maximum of 30°C. Do not dry clean. Do not iron. Do not use bleach. Do not tumble dry. Do not expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc.

Notes: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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