## Cutimed® Sorbact Hydro®

## en

Read the instructions for use before starting treatment with Cutimed® Sorbact Hydro®.

**Device Description**

Cutimed® Sorbact Hydro® is a non-absorbent bacteria and fungi binding wound dressing, based on Sorbact® Technology. It consists of a Sorbact® DACC™ coated wound contact layer covered with a hydrogel containing carbomer and propylene glycol (10%). Cutimed® Sorbact Hydro® treats and helps to prevent wound infections and facilitates the wound healing process. Sorbact® wound contact layer is shown to bind wound related pathogens up to seven days *in vitro*.

**Intended Purpose**

Cutimed® Sorbact Hydro® is intended for use in management of clean, contaminated, colonized or infected dry to exuding wounds, such as surgical wounds, traumatic wounds, burns, pressure ulcers, diabetic foot ulcers and leg ulcers. Cutimed® Sorbact Hydro® can be used on both superficial and deep wounds.

**Intended User and Use Environment**

Cutimed® Sorbact Hydro® is intended to be used on children and adults. Cutimed® Sorbact Hydro® is intended to be used by healthcare professionals, or by lay persons under supervision of a healthcare professional. The dressing is intended to be used in healthcare facilities and home environment.

**Mode of Action**

Cutimed® Sorbact Hydro® binds common wound microorganisms, such as *Staphylococcus aureus* (including MRSA), *Streptococcus* spp., *Escherichia coli*, *Pseudomonas aeruginosa* and *Candida albicans*, as shown *in vitro*. There is no release of antimicrobial agents to the wound.

**Clinical Benefits**

Use of Cutimed® Sorbact Hydro® can reduce bioburden and helps to prevent infection. The dressing helps reducing pain and can improve wound healing and decrease wound size. Cutimed® Sorbact Hydro® donates moisture, enables a moist environment and facilitates autolytic debridement.

**Contraindications**

There are no known contraindications to the use of Cutimed® Sorbact Hydro®. Warnings and precautions should be noted.

**Warnings and Precautions**

Do not re-use. Cutimed® Sorbact Hydro® is for single patient and single use only. Re-use may lead to cross contamination and increased risk of infection. Do not use if the peel pouch is already opened or damaged, since the sterility cannot then be guaranteed. Do not use on patients with a known allergy to propylene glycol or with a known hypersensitivity to the dressing (dialkylcarbamoyl chloride [DACC™], cellulose acetate, carbomer, propylene glycol and water). Caution should be taken when exposing children below 5 years and patients suffering from liver and kidney disease, due to limited capacity to metabolize propylene glycol. High exposure, e.g. in case of simultaneous use of other products containing propylene glycol, may in rare cases lead to adverse events, e.g. hypersomnolality or metabolic acidosis. Do not use in combination with fatty products, such as ointments, creams and solutions as they may decrease the binding of microorganisms. Do not re-sterilize. If the treated condition deteriorates, fails to improve or a side effect is observed, consult an appropriate clinician.

**Instructions for Use**

- Prepare the wound and surrounding skin according to local clinical practice.
- Select an appropriate dressing size.
- Remove the dressing from the peel pouch using an aseptic technique.
- If the dressing is cut, use an aseptic technique.
- Apply the dressing. Ensure that the dressing comes into direct contact with the complete wound surface to allow microorganisms to bind to the dressing. The dressing should not overlap the surrounding skin. Avoid dense packing when used in deep wounds.
- Apply a moisture preserving secondary dressing appropriate for the exudate level, such as foam dressing, absorbent pad or film dressing, and fixate.
- The dressing change frequency depends on exudate levels and the overall condition of the wound and surrounding skin.
- Apply the clinical condition allow, the dressing can be left in place for up to 7 days.
- Should the dressing adhere to the wound, moisten the dressing to assist removal and to avoid disruption of the healing wound.

Cutimed® Sorbact Hydro® can be used in conjunction with compression therapy. Cutimed® Sorbact Hydro® is MRI safe. Prior to radiation therapy, remove Cutimed® Sorbact Hydro® if placed in the radiation field. A new dressing can be applied following treatment.

**Storage and Disposal**

Cutimed® Sorbact Hydro® should be stored dry and kept away from sunlight. Disposal should be made according to local environmental procedures.

**Notice to User**

Any serious incident that has occurred in relation to the device should be reported to ABIGO Medical AB (complaints@abigo.com) and the competent authority of your state.

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|                                                                                    |                            |                                       |                                                                                                                                                                                                                                                                                                                                                                                                         |
|------------------------------------------------------------------------------------|----------------------------|---------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|   | Artwork ID:<br>JBR19125.01 | Review Loop:<br>05                    | Printing Colours:<br> Cyan<br> Magenta<br> Yellow<br> Black |
| Project Title:<br>C3132_GoPro Cutimed Sorbact Hydro_IFU ROW                        |                            | Date of Review Loop:<br>13.02.2024_RB |                                                                                                                                                                                                                                                                                                                                                                                                         |
| Contact<br>ArtworkManagementMedical@essity.com                                     |                            | Replaces Artwork ID:<br>JBR17647.01   |                                                                                                                                                                                                                                                                                                                                                                                                         |
| Contact Address: BSN medical GmbH<br>Schützenstraße 1–3<br>22761 Hamburg   Germany |                            | Technical Drawing for print ID<br>-   | Guiding Colours:<br> Diecut - DO NOT PRINT                                                                                                                                                                                                                                                                         |
| Languages:<br>EN FR ES IT NL PT PT (Brazil) EL (GR) SL ID AR CS SK                 |                            | Note                                  |                                                                                                                                                                                                                                                                                                                                                                                                         |

# ESKO WebCenter Approval Information

**Project:** JBR19125.01  
**Project Number:** C3132 Cutimed Sorbact Hydro  
**Approval Email:** Eva-Lie Essman:evalie.essman@essity.com,Micael Lagerqvist:micael.lagerqvist@essity.com, Inger Stenseke Larsson:inger.stenseke.larsson@essity.com  
**Approval Comment:** All below mentioned approvers attest to the accuracy and integrity of this document by signing the document with digital signature implemented in WebCenter.  
**Project Name:** GoPro Cutimed Sorbact Hydro  
**Reason for Change:** name change from CM Sorbact gel to CM Sorbact Hydro  
**Document:** JBR19125.01.pdf  
**Document description:**  
**Version:** 5  
**Version ID:** 00002\_0000034652  
**Downloaded:** Feb 20, 2024 at 9:49 AM GMT+01:00  
**By:** Hilke Meyer (DEHHIME) as Heiko Hess (DEHHEEE)  
**Approval state:**

## Approved

| St<br>ag<br>e<br>Nr | Approver                                                          | Role / Group             | Date and Time                     | Action                    | Comment |
|---------------------|-------------------------------------------------------------------|--------------------------|-----------------------------------|---------------------------|---------|
| 1                   | Eva-Lie Essman (SEHEVES), ESSITY - regular user (Germany)         | REGULATOR Y AFFAIRS ROLE | Feb 14, 2024 at 2:25 PM GMT+01:00 | Approved (Final approval) |         |
| 1                   | Lagerqvist, Micael (SEHMILG), ESSITY - regular user (Germany)     | PACKAGING ROLE           | Feb 15, 2024 at 7:38 AM GMT+01:00 | Approved (Final approval) |         |
| 1                   | Inger Stenseke Larsson (SEHINST), ESSITY - regular user (Germany) | MARKETING ROLE           | Feb 16, 2024 at 8:03 AM GMT+01:00 | Approved (Final approval) |         |