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





## ASSEMBLY INSTRUCTIONS

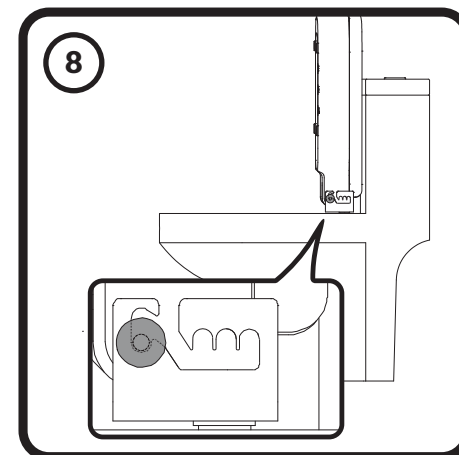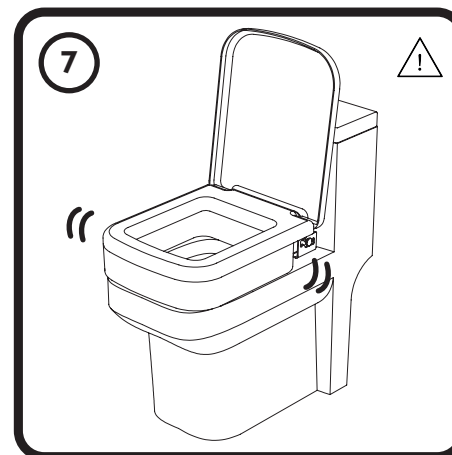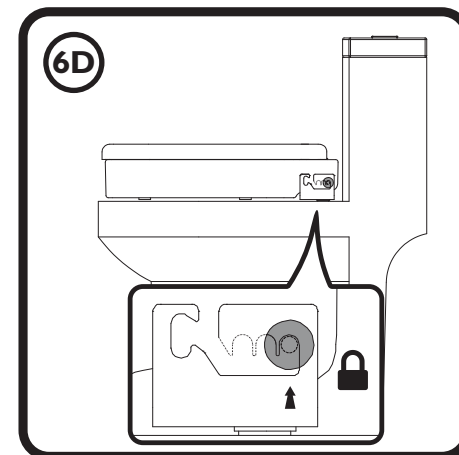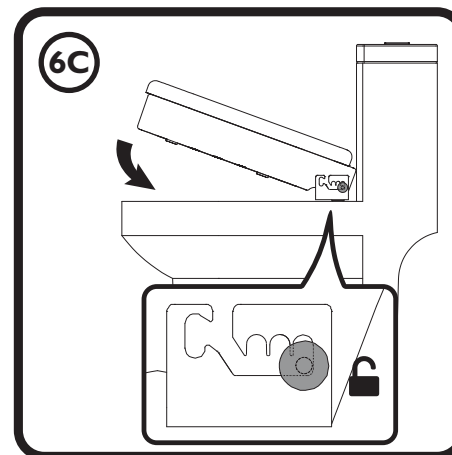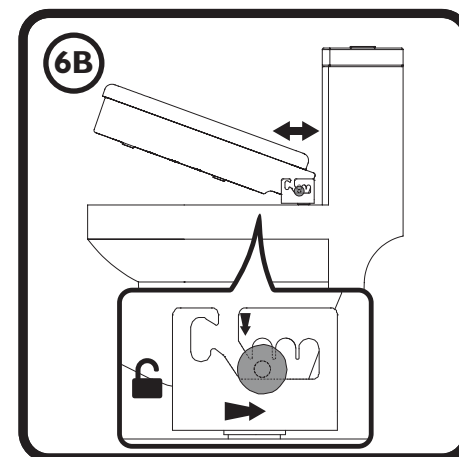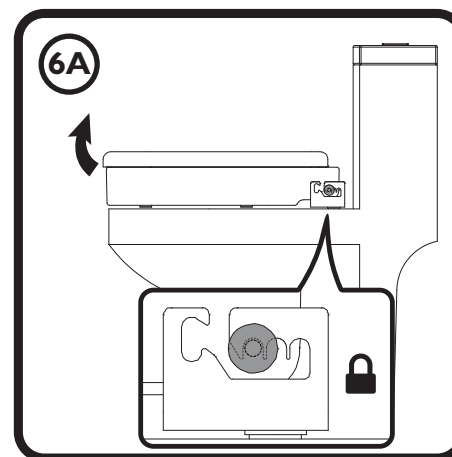**ASSEMBLY INSTRUCTIONS:** **1.** REMOVE EXISTING TOILET SEAT AND FIXINGS. **2.** INSERT THE BUNGS **[a]**, ENSURING THEY SIT FLUSH. **3.** PLACE AND ALIGN THE METAL BRACKET **[b]**. SECURE USING THE METAL WASHER **[c]** AND SCREW **[d]**. TIGHTEN UNTIL A SECURE FIT IS REACHED. ENSURE THE BRACKET SITS CENTRALLY BETWEEN FIXING POINTS. **4.** FASTEN YOUR EXISTING TOILET SEAT TO THE "ASHBY LIFT" **[e]** USING EXISTING FIXINGS – MAXIMUM ALLOWABLE THREAD LENGTH FOR FIXINGS IS 6.5CM. IF THE THREAD LENGTH IS LONGER, NEW FIXINGS WILL NEED TO BE SOURCED. **5.** INSERT THE "ASHBY LIFT" INTO THE BRACKET AND SLIDE TO A SUITABLE SLOT POSITION. POSITION MUST BE THE SAME ON EACH SIDE. **6A.** TO ADJUST THE POSITION, PIVOT THE SEAT. **6B.** IN ITS UNLOCKED POSITION THE SEAT CAN NOW BE MOVED. **6C.** LOWER THE SEAT. **6D.** ENSURE THE SEAT IS LOCKED AND LOCATION POINTS ARE THE SAME ACROSS BOTH SIDES. **7.** CHECK FOR STABILITY BEFORE EACH USE. **8.** THE SEAT CAN BE EASILY REMOVED OR FULLY PIVOTED FOR CLEANING. TO FULLY PIVOT, RAISE SLIGHTLY TO UNLOCK AND POSITION IN THE FRONTMOST POSITION.

ENSURE FIXINGS ARE TIGHT BEFORE USE – CHECK PERIODICALLY. ENSURE FOUR NON-SLIP FEET ARE IN PLACE ON THE LIFT BEFORE USE. IT IS THE RESPONSIBILITY OF THE USER/ OWNER TO CARRY OUT AN INITIAL AND REGULAR LOCAL RISK ASSESSMENTS.

- ⊕ = MAXIMUM USER WEIGHT.
- = WIPE WITH A NON-ABRASIVE CLOTH AND SOAPY WATER.

ANY **SERIOUS ACCIDENT** OCCURRING THROUGH USE SHOULD BE REPORTED TO THE **EC REP** (EU CUSTOMERS) OR THE **MANUFACTURER** (UK & NON-EU CUSTOMERS)

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