Askina® Carbosorb

cs	Krytí s aktivním uhlím s vynikajícími absorpčními účinky
de	Superabsorbierende Aktivkohlewundauflage
en	Activated Charcoal Super Absorbent Dressing
es	Apósito superabsorbente de carbón activado
et	Aktiveeritud sõega üliimav side
fi	Superimukykyinen aktiivihiilisidos
fr	Pansement super absorbant au charbon actif
it	Medicazione superassorbente al carbone attivo
nl	Superabsorberend verband met actieve kool
pt	Penso Superabsorvente com Carvão Ativado
sv	Superabsorberande förband med aktivt kol

B | BRAUN

Askina® Carbosorb

	Winner Medical Co. Ltd. Winner Industrial Park, No 660 Bulong Road, Longhua District, 518109 Shenzhen, China	 0123
	Shanghai International Holding Corp. GmbH (Europe) Eiffestraße 80, 20537 Hamburg, Germany	
	B. Braun Avitum AG Schwarzenberger Weg 73-79 34212 Melsungen, Germany	
		
		
	08/22	

en	INSTRUCTION FOR USE
Product description Askina® Carbosorb is an absorbent dressing which provides exudate control and odour adsorption. The structure makes it a suitable dressing for the moderate to heavily exuding wounds. The dressing is in a multilayer construction. It consists of a white wound contact layer that does not adhere to the wound bed; The inside layer includes a superabsorbent pad layer that provides liquid absorption and retention; and an activated charcoal cloth layer that adsorbs wound malodour. There is a blue fluid repellent backing layer which can reduce fluid strikethrough, its vapour permeable qualities improves breathability and exudate management.	

Indications
Activated charcoal super absorbent dressing is indicated for malodorous wounds with moderate to heavy exudate, such as diabetic foot ulcer, pressure ulcer, leg venous and arterial ulcer, first and second-degree burns, post-operative wounds and traumatic wounds. This is a professional use only and single use device. The dressing change interval may be several days. Change the dressing before it is fully saturated, at signs of leakage, or as indicated by actual clinical use. The same dressing cannot be used for more than 7 days, continuous use of the product cannot be more than 30 days. The treatment can continue for 30 days, within that period the dressing can be replaced, however the total treatment time should not exceed 30 days.

Precautions
1) If there are signs of infection consult a health care professional. 2) Do not reuse. Reuse will cause cross-contamination. 3) Do not use if package is damaged or open.

Contraindications
1) Do not use as a primary treatment on bleeding and dry wounds. 2) Do not use in cavity wounds since the dressing will expand when absorb liquid.

Instruction for use
1. Select a size of dressing enough to overlap the wound edges at least 2-3cm. 2. Gently apply the dressing directly on the wound site with the white hydrophilic side of dressing onto the wound surface. 3. The dressing shall be fixed by an appropriate secondary dressing, such as bandage or tape.

Frequency of change
The dressing can be left in place for up to 7 days. But wearing time will depend on the level of exudate. Because of the high absorption capability of the dressing, it may exert pressure against the wound bed and require changing.

Warnings
1) Do not stretch the dressing. 2) Do not use on the patients with a known hypersensitivity to the product itself or to its components.

Storage and transport conditions
1. The dressing should be stored in warehouse under normal room condition, temperature and humidity of room will not be specified strictly but require cool and dry conditions. Direct sunlight shall be avoided. 2. The transportation should be to keep away from direct sunlight and keep dry.

Disposal
Disposal should be handled according to local environment procedures.

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