Ref: RPKK102UN, RPKK103UN

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Work table CLASSIC

Assembling:

Based on the width of chassis TOM 5, FROGGO adjust the brackets in the groove to optimal position for your type and size of the stroller. Loosen the nuts (1) on the groove of brackets and move with it as is needed.

Then tighten the nuts. During installation it is necessary to observe the correct positioning of both sides of the brackets.

Horizontal adjustment: By enabling rosettes set shift of the table and then tighten it.

Mounting:

Take off the front safety bar and place the tubes of work table into the holes instead of the safety bar – secure with spring fuse.

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Warranty conditions:

- On this product is applied 24 months warranty from the date of sale. The warranty is applied for material and manufacturing defects.
- The product has to be used entirely for the purpose it is designed for and the instructions in the user guide must be followed.
- The product has to be properly used, maintained, cleaned and stored in accordance to the user guide instructions.**
- Do not pull down the identification tag. It has to be readable and undamaged for the whole warranty period.
- The warranty case has to be claimed by the seller, where the product was bought. Properly filled “Warranty sheet” and the proof of purchase must be declared by the warranty application. Every defect should be claimed immediately
- The claim procedure can be started when the device is delivered complete, in original packaging, carefully packed and secured against damage. The owner is committed to deliver the device clean in accordance to the general hygienic measures. The basic maintenance and cleaning cannot be expected or even required when claimed.
- The warranty does not cover: failures caused by the overloading of the products, accident, forced damage, unauthorized changes made on the product and/or its accessories, natural wear and tear of parts, especially upholstery dirt and damage, wear of plastic parts, brakes, tyre profile and tyre puncture, etc.
- The warranty is terminated whenever the following conditions arise: the user fails to follow the instruction mentioned in this user guide and manufacturer’s storage and maintenance suggested procedures, or whenever the product is serviced by any other party except of authorised service centres, or when the identification label is not stucked on the stroller and also in the warranty sheet.
- The warranty period cannot be interchanged with using period, that means for the lifetime period when correct use and maintenance of the product in accordance to the user guide are fulfilled.
- The producer nor the importer is not liable for any losses and injuries caused by manipulation and usage of the product in conflict with the “User Guide” instructions.

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