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Product Description
 3M™ Nu-Gel™ Hydrogel with Alginat is a transparent hydro active amorphous gel containing sodium alginate. The hydrogel creates a moist wound healing environment which assists with natural autolytic debridement whilst the alginate component enhances its absorptive capabilities. The gel can be used to soften and hydrate eschar by facilitating rehydration of the wound.

Intended User
 For use by medical professionals or under the guidance of a medical professional.

Indications
 Nu-Gel Hydrogel is indicated for debridement and desloughing of wounds together with the management of chronic wounds throughout all stages of the healing process.
 [Deviation for Spanish: Nu-GEL Hydrogel is indicated for the treatment of decubitus ulcers and varicose ulcers in all stages of healing.]

Precautions
 If wound infection develops, appropriate antimicrobial therapy should be initiated. The use of Nu-Gel Hydrogel may be continued under medical supervision.

DIRECTIONS FOR USE
Application - see diagram, also on back of ampoule label

1. Carefully cleanse the wound.
2. Remove the label from the concertina pack by pulling the label as indicated.
3. Remove the cap, and swab the nozzle with a suitable antiseptic swab (1-2).
4. Twist to remove the tab (3).
5. Nu-Gel Hydrogel can be applied to the wound with a simple one-handed technique (4).

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