3M™ Adaptic™ Non-Adhering Dressing
Product Description
3M™ Adaptic™ Non-Adhering Dressing is a primary dressing made of knitted cellulose acetate fabric and impregnated with a specially formulated petroleum emulsion. It is designed to help protect the wound while preventing the dressing from adhering to the wound.

- Helps prevent dressing adherence
- Helps protect regenerating tissue and minimizes patient pain and trauma during dressing changes
- Helps prevent the pooling of fluid at the wound site
- Can be cut to wound size without unraveling and shredding

Indications
Adaptic non-adhering dressing is indicated for dry to highly exuding wounds where adherence of dressing and exudate to be prevented including:

- 1st and 2nd degree burns
- lacerations and abrasions
- grafts
- nail extractions
- ulcerous pressure ulcers
- eczema
- staples and suture line
- surgical incisions
- reconstructive procedures

Intended User

For medical professionals or under the guidance of a medical professional.

DIRECTIONS FOR USE

Skin Preparation

1. Prepare the wound according to wound management protocol
2. Ensure skin surrounding the wound is dry.

Dressing Application

1. Carefully clean the wound following facility protocol.
2. Cut Adaptic non-adhering dressing to size if required and place directly on the wound.
3. Cover with sterile secondary dressing of your choice depending on the amount of wound exudate.

4. Adaptic non-adhering dressing may be left in place for several days depending upon amount of exudate.

Note: Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or relevant regulatory authority.

Do not re-use. Reuse may lead to infection or other illness/injury.

Federal Law (USA) restricts this device to sale by or on the order of a properly licensed healthcare practitioner. This caution is not applicable outside the USA.

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